making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - Polski
- FAQ - 中文
- FAQ - 日本語
- FAQ - 한국어
- FAQ - اللغة العربية

# Basic quattro IS

FAQ - frequently asked questions

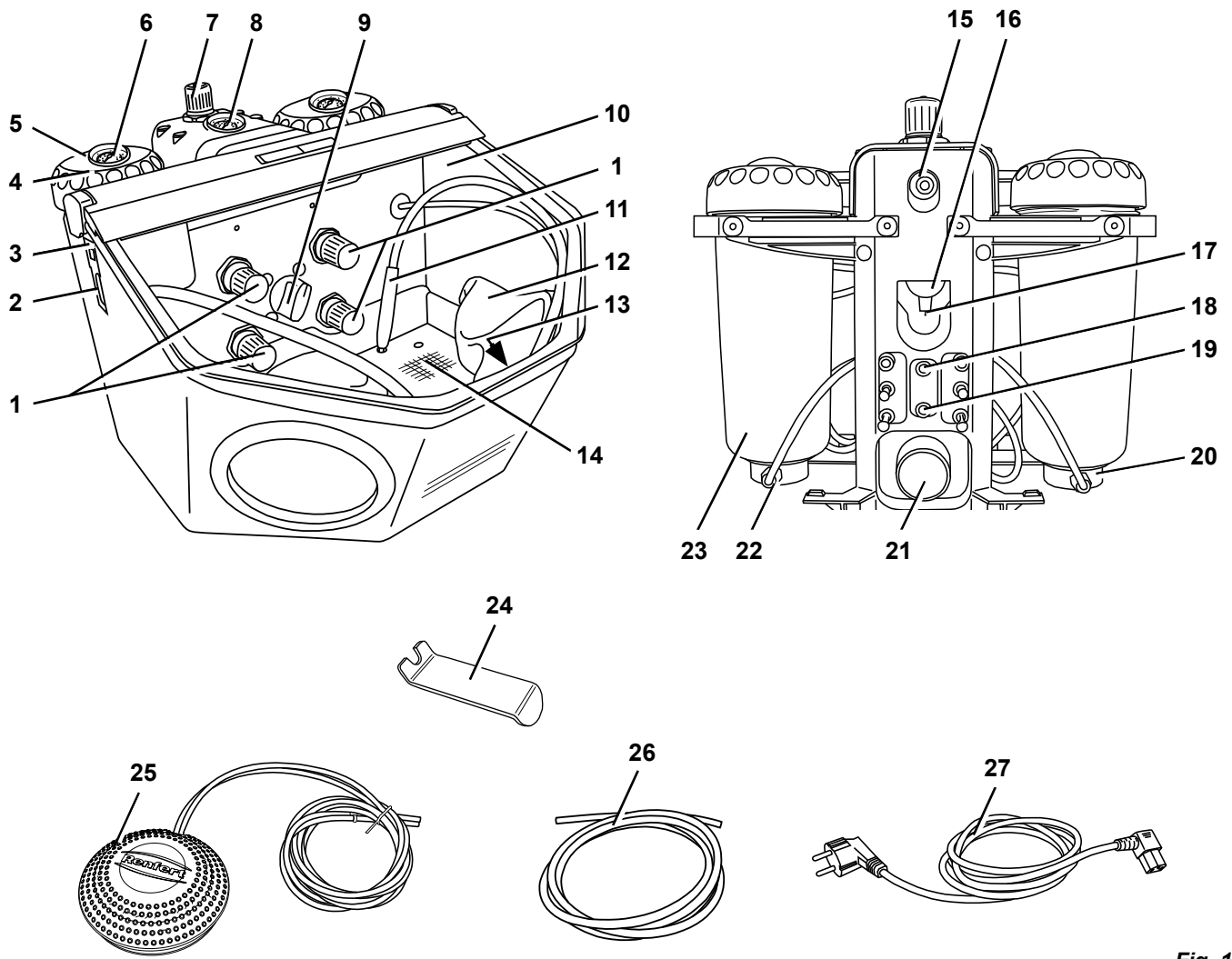
Fig. 1

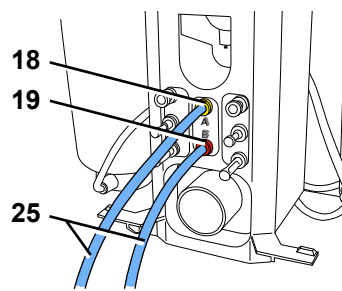
Fig. 2

# FAQ - Deutsch

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# FAQ - English

For information on operating elements, see [Fig. 1](#) and [Fig. 2](#) on page 2 of this document.

For chapter indications, refer to the complete instruction manual on the internet.

Problem	Possible cause	Corrective action
<b>No air and no abrasive.</b>	<ul style="list-style-type: none"> <li>Blasting nozzle blocked at the hand-piece.</li> <li>Foot switch connecting lines kinked.</li> <li>Tank selection switch in an intermediate position.</li> </ul>	<ul style="list-style-type: none"> <li>Unscrew the nozzle from the hand-piece and blow it out.</li> <li>With the nozzle removed, blow out the blasting hose by activating the foot pedal.</li> <li>Inspect / correct the connection lines.</li> <li>Turn the switch until you feel it click into place.</li> </ul>
<b>Only air, no or very little abrasive.</b>	<ul style="list-style-type: none"> <li>The dosage nozzle in the mixing chamber is blocked due to contaminated or moist sand.</li> </ul>	<ul style="list-style-type: none"> <li>Cleaning the dosing nozzle, see chapter 6.4.</li> </ul>
<b>Only air, no or very little abrasive.</b>	<ul style="list-style-type: none"> <li>Mixing chamber filter combination blocked.</li> <li>Abrasive contains impurities or is moist.</li> </ul>	<ul style="list-style-type: none"> <li>Change filter cartridge (<a href="#">22 / Fig. 1</a>) (wrench SW14).</li> <li>Replace the abrasive.</li> </ul>
<b>Unit blasts continuously.</b>	<ul style="list-style-type: none"> <li>Foot switch connected incorrectly.</li> <li>Valve in foot switch is defective.</li> </ul>	<ul style="list-style-type: none"> <li>Inspect / correct the foot switch connection.</li> <li>Replace foot switch.</li> </ul>
<b>Lamp is not alight.</b>	<ul style="list-style-type: none"> <li>Power cord not plugged in.</li> <li>Illumination defect.</li> </ul>	<ul style="list-style-type: none"> <li>Plug the power cord into the wall outlet.</li> <li>Change light using the service or see spare part list.</li> </ul>
<b>Compressed air leaking from the foot switch.</b>	<ul style="list-style-type: none"> <li>Pneumatic hoses reversed.</li> </ul>	<ul style="list-style-type: none"> <li>Reverse the connections.</li> </ul>
<b>Foot switch fails to switch on.</b>	<ul style="list-style-type: none"> <li>No pressure at the compressed air connection.</li> <li>Pneumatic hoses are kinked.</li> </ul>	<ul style="list-style-type: none"> <li>Inspect the compressed air lines and / or the compressor.</li> <li>Reposition the hoses.</li> </ul>
<b>Foot switch hoses cannot be disconnected from the unit.</b>	<ul style="list-style-type: none"> <li>Push-in connection is not released (<a href="#">Fig. 2</a>).</li> </ul>	<ul style="list-style-type: none"> <li>Press the push-in connection ring on the hose side firmly while pulling on the hoses. Please use the release tools supplied (<a href="#">24</a>).</li> </ul>
<b>Quick-stop not functioning (the nozzle blasts continuously).</b>	<ul style="list-style-type: none"> <li>Pneumatic connections are loose.</li> <li>Sandblasting hose worn.</li> <li>The connecting pressure at the external air connection (<a href="#">15, Fig. 1</a>) is too small.</li> <li>Outer pressure regulator on the maintenance unit is altered (fixing screw is loosen).</li> </ul>	<ul style="list-style-type: none"> <li>Inspect the pneumatic connections for correct seating.</li> <li>Replace the sandblasting hose.</li> <li>Increase the external connecting pressure at a min. of 5 bar. The external connecting pressure is displayed on the manometer (<a href="#">8, Fig. 1</a>).</li> <li>Turn the inner pressure regulator clockwise until stop. Activate foot pedal and set the pressure regulator on the maintenance unit at 5 bar (see manometer).</li> </ul>
<b>Sound of rushing air when the unit is operating.</b>	<ul style="list-style-type: none"> <li>A small amount of leakage is always present at the control valves, thus allowing (a minimum) of air to escape.</li> <li>Continue to turn the tank selection switch whilst activating the foot switch at the same time.</li> </ul>	<ul style="list-style-type: none"> <li>This is not a fault, but instead is the result of a pressure release opening which must not be blocked.</li> <li>Only turn the tank selection switch when the foot switch is not activated.</li> </ul>

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