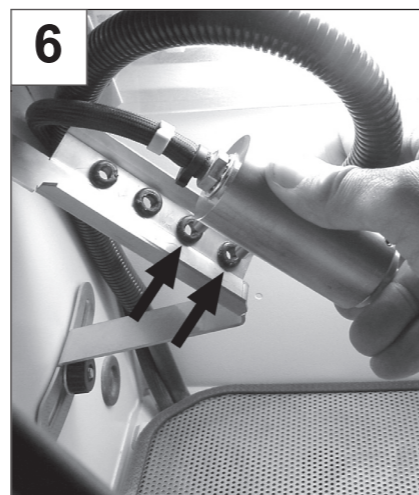
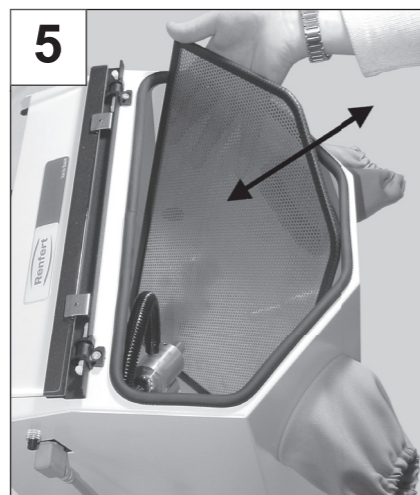
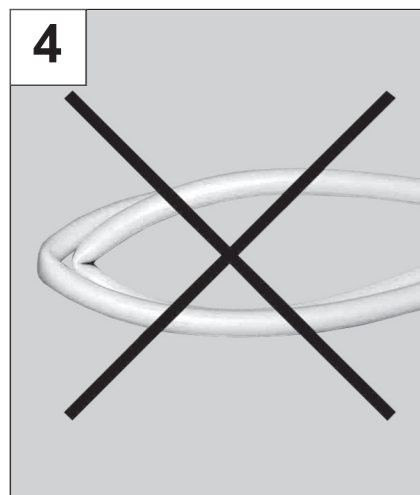
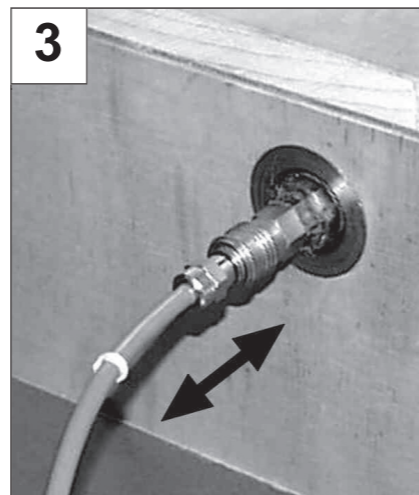
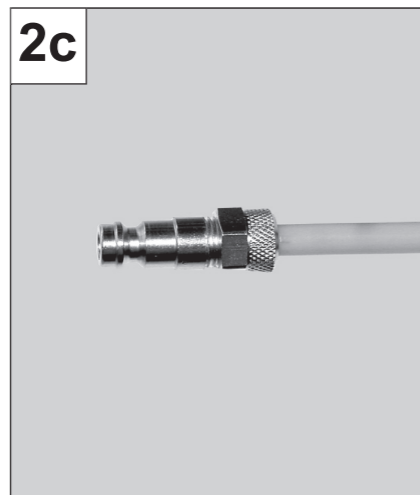
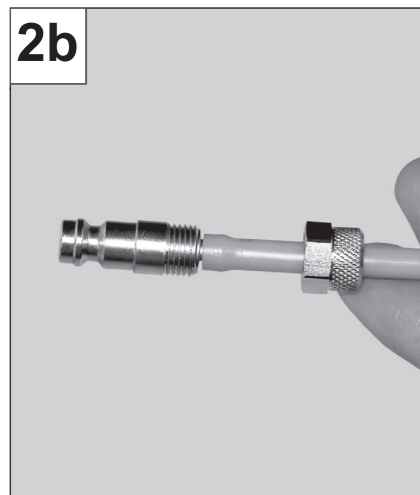
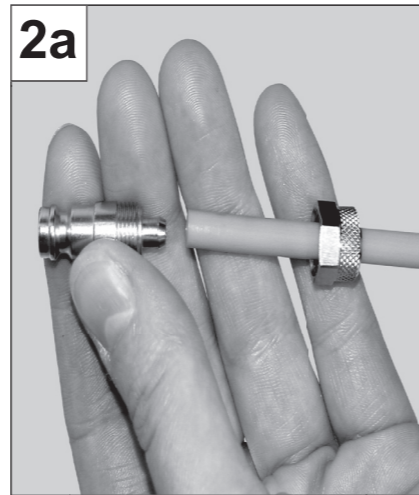
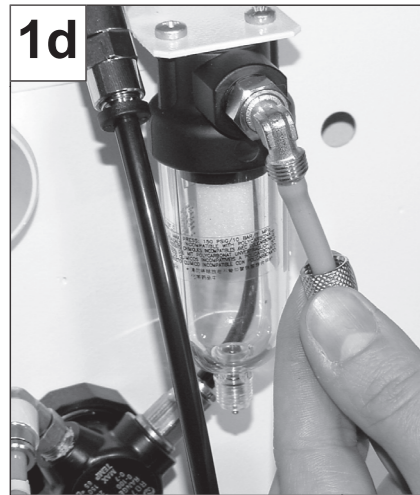
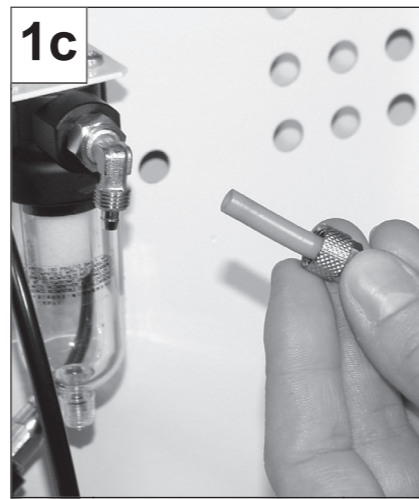
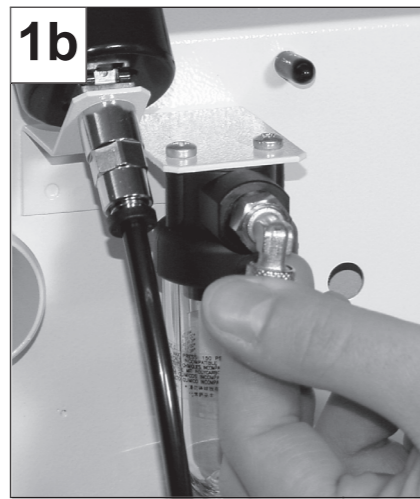
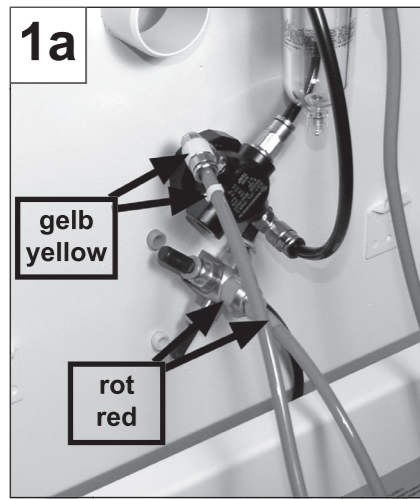
Vario basic

No. 2960-0000 / 2960-1000

Bedienungsanleitung
Instruction manual • Mode d'emploi • Istruzioni d'uso
Instrucciones para el servicio • Gebruiksaanwijzing
Instrukcja obsługi • Инструкция по эксплуатации
操作手冊 • 取扱説明書 • إرشادات التشغيل

Seriennummer, Herstelldatum und Geräte-Version befinden sich auf dem Geräte-Typenschild.
 Serial number, date of manufacturing and unit version are shown on the type plate of the unit.
 Le numéro de série, la date de fabrication et la version se trouvent sur la plaque signalétique de l'appareil.
 Il numero di serie, la data di costruzione e la versione si trovano sulla targhetta dell'apparecchio.
 El número de serie, la fecha de fabricación y la versión del aparato están indicados en la placa identificadora del aparato.
 Seriennummer, produktiedatum en versie staan op het apparaat-type plaatje.

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ENGLISH

1. Introduction

We are pleased with your decision to purchase the *Vario basic*.

This unit sets new standards with regard to multifunctionality, performance, and ergonomics.

Please read the following operating instructions carefully and observe all the information they contain in order to ensure long and trouble-free service.

2. Application Area

The *Vario basic* recyclable blaster is used in dental laboratories to remove investments and oxides from cast parts (primarily model castings).

Optionally, the unit can be expanded by up to two micro sandblasting tanks.

Thus the application area also includes the removal of investments and oxides from crowns / bridges and inlays as well as surface treatment by polishing.

The device is delivered without micro sandblasting tanks but is fully ready to be outfitted with 1 or 2 micro sandblasting tanks. Further, an additional device can also be connected. Device selection is by means of the multifunction switch (selector switch) inside the blasting chamber.

If micro sandblasting tanks are to be installed from the outset, this should be done prior to the unit being filled with abrasive.

2.1 Ambient Conditions

(in accordance with DIN EN 61010-1)

The *Vario basic* may only be operated:

- Indoors;
- Up to an altitude of 2,000 m [6,500 ft] above sea level;
- At an ambient temperature range of 5 - 40°C [41 - 104°F];
- At a maximum relative humidity of 80% at 31°C [87.8°F], dropping linearly to 50% relative humidity at 40°C [104°F];
- With a mains power supply whose voltage fluctuations do not exceed 10% of the rated value;
- Under contamination level 2 conditions;
- Under over-voltage category II conditions.

*) Between 5 - 30°C [41 - 86°F] the unit can be operated at a relative humidity of up to 80%. At temperatures between 31 - 40°C [87.8 - 104°F] the humidity must decrease proportionally in order to ensure proper operability (e.g., at 35°C [95°F] = 65% relative humidity, at 40°C [104°F] = 50% relative humidity). The unit may not be operated at temperatures above 40°C [104°F].

3. Hazard Information

3.1 Employed Symbols

The following symbols are used in these instructions and on the unit:

Danger

This indicates an imminent risk of injury.

Electrical hazard

There is a risk of electrical shock.

Attention

Failure to observe the associated information can result in damage to the unit.

Note

The operating instructions contain useful information to make handling easier.

Wear eye protection.

Wear protective gloves.

For indoor use only.

Note the operating instructions.

3.2 Hazard Information

- The *Vario basic* is an electrical device with inherent hazard potential. This device may only be installed and operated by authorized specialists after examination of compliance with locally applicable standards.
- Prior to commissioning, compare the information on the nameplate with the specifications of the local mains power network and the connection values for compressed air.
- In order to avoid health hazards, never operate the sandblasting unit without adequate dust extraction and proper personal protection. The type of extraction employed is dependent on the type of dust being generated. In this regard, please note the requirements of EN 60335-2-69, Attachment AA, or ask your professional organization or regional authority.
- Improper operation can result in the risk of eye and skin injuries.

- Always wear appropriate eye protection when operating the unit! Failure to wear eye protection can result in eye injuries due to airborne particles.
- Never direct the nozzles towards your eyes or uncovered areas of skin!
- Never work with the viewing pane open!
- Regularly inspect all connecting lines and hoses (e.g., power cord and compressed air hoses) for damage (e.g., kinks, cracks, porosity).
- Units with damaged connecting lines or hoses, or those which exhibit other defects may not be used.
- Always disconnect the unit from the compressed air supply prior to performing any maintenance tasks.
- Always disconnect the unit from the mains power supply prior to performing any work on electrical components.
- Do not activate the foot switch when only the compressed air inlet hose is connected. The unconnected hose can whip about, resulting in a significant risk of injury.
- Always disconnect the unit from the mains power supply prior to replacing the fluorescent tube.
- Never press on or bend the lamp body during replacement (breakage risk). If necessary, wear gloves or use a cloth.

When using micro sandblasting tanks:

- Check the tank cover for a secure seat prior to starting operation. Tank covers that are not securely closed can explosively open. Ejected parts and abrasive represent a significant risk of injury.
- Always wear safety glasses when working with the tank (filling, cleaning, maintenance).
- Abrasive residue on the seal will result in leaks and premature seal wear. Always clean the threads and seal after filling and handtighten the cover.
- Solvents and tensides can result in the formation of microscopic cracks in the plastic (explosion hazard!). Use only a clean, dry cloth to wipe the tank out and off. Never write on the tanks or place adhesive labels on them.
- Regularly inspect the tanks and tank covers for damage and immediately replace any suspect components.

3.3 Liability Exclusion

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- The product is employed for any purposes other than those cited in the operating instructions;
- The product is altered in any way other than those alterations described in the operating instructions;
- The product is repaired by other than an

authorized facility or if any but Renfert OEM parts are employed;

- The product continues to be employed, despite obvious safety faults or damage;
- The product is subjected to mechanical impacts or is dropped.

4. Commissioning

4.1 Wall Mounting

The holder required for wall mounting can be ordered separately as an optional accessory (refer to the accessories list). The holder includes instructions for mounting the unit on a wall.

4.2 Setup and Connection

1. Remove the foot switch stored in the blasting chamber.
2. Connect the foot switch to the sandblasting unit by completely inserting the two red and yellow pneumatic hoses into their correspondingly marked connectors until they reach their stops (Figure 1a).
3. Connect the compressed air supply to the sandblasting unit:
 - Loosen the air connection's knurled sleeve nut on the unit (Figure 1b).
 - Push the knurled sleeve nut over the included blue compressed air connecting hose (Figure 1c).
 - Push the compressed air connecting hose onto the compressed air connection and tighten the knurled sleeve nut (Figure 1d, 1e).
4. Connect the compressed air hose to the compressed air supply:
 - Loosen the compressed air connector's knurled sleeve nut and push it over the included blue compressed air connecting hose (Figure 2a).
 - Push the compressed air connecting hose onto the compressed air connection and tighten the knurled sleeve nut (Figure 2b, 2c).
 - Connect the connector to the laboratory compressed air supply (Figure 3).

! Do not bend or kink the hose (Figure 4)!

4.3 Installing the Nozzle

1. Remove the base grate (Figure 5).
2. Insert the blasting nozzle into the retainer (Figure 6).
3. Check the extraction hose for proper seat (Figure 7).

4.4 Adjusting the Operating Pressure

1. Set the selector switch to the recycling sandblasting position (Figure 8, Position 1).
2. Adjust the blasting pressure to 1 - 6 bar [14.5 - 87 psi] (Figure 9). The current pressure setting is continuously displayed on the manometer.

4.5 Adding Abrasive

1. Remove the base grate (Figure 5).
2. Add abrasive (Figure 10). Approx. 5 kg (+/- 2 kg)
3. Reinsert the base grate (Figure 5).

i Use only clean, dry abrasive with the appropriate grain size (refer to the accessories list). We recommend an abrasive with a grain size of 125 - 250 µm [115 - 60 mesh].

! Max. grain size: 500 µm [32 mesh].

4.6 Installing the Fluorescent Lamp

⚠ The unit must still be disconnected from the mains power supply!

1. Insert the fluorescent lamp horizontally (Figure 11).
2. Remove the foil strip from both ends of the lamp cover (Figure 12).
3. Secure the lamp cover with two knurled nuts (Figure 13).

4.7 Starting the Unit Illumination

1. Connect one end of the power cord to the unit (Figure 14) and plug the other end into the wall outlet (Figure 15).
2. Switch the lamp on (Figure 16).

4.8 Connection to the Dust Extractor

Push the extraction hose onto the extractor connection (Figure 17-a). An extraction hose is not included in the standard *Vario Basic* delivery package.

5. Operation

5.1 Device Selection

The *Vario basic* is delivered without micro sandblasting tanks but is fully ready to be outfitted with up to 2 micro sandblasting tanks. Further, an additional device operating with compressed air can also be connected to the unit.

Device selection (recyclable sandblaster, left or right sandblasting tank, additional compressed air device) is made using the selector switch in the blasting chamber (Figure 8):

- (1) Recyclable blasting;
- (2) Left micro sandblasting tank (optional);
- (3) Right micro sandblasting tank (optional);
- (4) Additional compressed air device (optional).

Sandblasting continues as long as the foot switch is depressed (Figure 18). A small amount of abrasive continues to flow once the foot switch is released.

Please use the provided gloves when sandblasting.

The cloth hand port cuffs do not need to be removed.

Do not operate the unit if the viewing pane is open.

5.2 Positioning the Blasting Nozzle

Nozzle distance and nozzle angle can be adjusted separately (Figure 19).

5.2.1 ADJUSTING THE NOZZLE DISTANCE:

Three different nozzle inserts are available to provide three different nozzle distances (Figure 19-a + Figure 6).

5.2.2 ADJUSTING THE BLASTING ANGLE:

1. Loosen the upper and lower knurled nuts on the retainer (Figure 19-b).
2. Adjust the nozzle to the desired angle.
3. Retighten the screw (Figure 19-b).

5.3 Adjusting the Extraction Power

If extraction power is insufficient, you can increase it by covering the opening on the back of the unit with a piece of tape (Figure 17-b).

6. Connecting Additional Tanks and Other Compressed Air Devices

6.1 Installing Micro Sandblasting Tanks

The installation of the additional tanks is described in a separate set of instructions included with the tanks.

6.2 Connecting Other Devices

The various devices are connected to the connectors on the back of the unit (Figure 20):

- 1 Recyclable sandblasting connection (pre-connected at delivery);
- 2 Left micro sandblasting tank connection (remove the plug before connecting);
- 3 Right micro sandblasting tank connection (remove the plug before connecting);
- 4 Connection for an additional compressed air device (remove the plug before connecting).

6.3 Filling the Micro Sandblasting Tanks

 **Never activate the foot switch while filling the tanks.**

 **Regardless of the pressure indicated on the manometer, the micro sandblasting tanks are only under pressure if the foot switch is activated.**

1. Unscrew the tank cover (Figure 21).
2. Add abrasive only to the max. fill mark (the point at which the tank jacket thickens below the cover thread) (Figure 22).
3. Reinstall the tank cover (Figure 21).

 **Before closing the tank, always clean any abrasive residue from the threads on the cover and on the tank.**

Do not misalign the cover.

 **Use only clean, dry abrasive with the appropriate grain size (refer to the accessories list).**

6.4 Connecting Additional Compressed Air Devices

Additional compressed air devices such as the *basic mobil* (No. 2914-xxxx) or *Power-Pillo* (No. 5022-3000) are connected to the back of the *Vario basic* (Figure 24, Section 6.2, Figure 20).

The connected device's compressed air hose can be inserted through the socket on the side (Figure 23).

Figure 24: Example of connecting a *basic mobil*.

7. Cleaning / Maintenance

  **Always disconnect the unit from the mains power and compressed air supplies prior to performing any cleaning and maintenance tasks.**

7.1 Cleaning the Interior

- Use only solvent-free cleansers (e.g., soapy water).
- Remove the base grate (Figure 5) and vacuum out the blasting chamber.
- Using a dry, lint-free cloth, wipe out the insides of the tanks and the tank covers!

7.2 Inspecting the Water Separator

- Drain any condensation water through the outlet valve.
- Press up on the valve from below (Figure 25).

7.3 Blasting Nozzle Replacement

- Loosen the setscrew (Figure 26) >>> Allen key included in the accessories kit.
- Pull the nozzle out (Figure 27).
- Inspect the mixing chamber for wear and remove any abrasive residue (Figure 28).
- Insert the new nozzle (Figure 27).
- Retighten the setscrew (Figure 26).

7.4 Changing the Abrasive

- Pull the unit out over the edge of the table (Figure 29).

 **Tipping hazard!**
A second individual should help steady the unit.

- Position the container to catch the used abrasive (Figure 30).
- Open the abrasive drain hole (Figure 31).
- Clean / and re-secure the filter (Figure 32 / Figure 7).
- Close the abrasive drain hole (Figure 31).
- Add new abrasive (Figure 10).

7.5 Replacing the Viewing Pane

- Loosen the retaining screws (Figure 33).
- Remove the old pane and install the new one (Figure 34).
- Carefully retighten the retaining screws (Figure 33).

7.6 Replacing the Blasting Hoses

Regularly inspect the blasting hoses for damage and wear. Replace damaged hoses immediately.

 **Disconnect the sandblasting unit from the compressed air supply when replacing the blasting hoses (Figure 3).**

- Drain the abrasive out of the unit (refer to Section 7.4).
- Loosen the hose clamp and pull the hose off the connector (Figure 35, 36, 37).
- Install new hoses (Figure 35, 36, 37).

 **Use only Renfert OEM replacement hoses!**

7.7 Replacing the Fluorescent Lamp

 **Disconnect the unit from the mains power supply before changing the fluorescent lamp (Figure 15)!**

 **Never press on or bend the lamp body during replacement (breakage risk). If necessary, wear gloves or use a cloth.**

- Loosen the knurled nuts (Figure 13) and remove the cover.
- Pull the fluorescent lamp horizontally out of its socket (Figure 11).
- If possible, vacuum out the socket.
- Insert the fluorescent lamp horizontally into the socket (Figure 11).
- Reinstall the cover and secure it with the two knurled nuts (Figure 13).

! Use only Renfert OEM fluorescent lamps (refer to the enclosed spare parts list).

8. Spare Parts

Please refer to the enclosed spare parts list for the numbers of all consumables and spare / replacement parts.

9. Warranty

Provided the unit is properly used, Renfert **warrants** all components of the *Vario basic* for a period of **3 years**. The original sales receipt from the dealer must be provided when making warranty claims. Components subject to natural wear (e.g., nozzles, hoses, fluorescent lamps, connectors for blasting hoses, view panes, filter materials, etc.) are excluded from this warranty. The warranty is voided in case of improper use; failure to observe the operating, cleaning, maintenance, and connection instructions; in case of independent repairs or repairs by unauthorized personnel; if spare parts from other manufacturers are employed, or; in case of unusual influences or influences not in compliance with the utilization instructions. Warranty service shall not extend the original warranty.

10. Technical Specifications

Mains voltage:	230 V, 50/60 Hz 120 V, 50/60 Hz
Power consumption:	12.3 W / 9 W (230/120 V)
Power cord length:	2 m [78.74 inches]
Connecting pressure, external:	6 - 10 bar [87 - 145 psi]
Operating pressure:	1 - 6 bar [14.5 - 87 psi]
Air consumption:	200 l/min. at 6 bar [87 psi]
Ø, hose connection:	Internal: 4 mm [0.16 inches] External: 6 mm [0.24 inches]
Dimensions (WxHxD):	380 x 540 x 450 mm [14.92 x 21.28 x 17.73 inches] (with 2 micro sandblasting tanks connected)
Weight (empty):	approx. 14 kg
Ø, pipe union for external extraction:	
Internal:	35 mm [1.38 inch]
External:	40 mm [1.58 inch]
Blasting chamber volume:	23 l
Sound level in accordance with DIN 45635-01-KL3:	< 70 dB(A)

11. Notes Regarding Return Shipment

- Request Renfert shipping boxes (Tel. +49 (0)7731 8208-61).
- Empty all blasting tanks.

! If a Renfert shipping box is not used, always remove the additional tanks (otherwise we can assume no liability for damage during shipping).

The proper return shipment form can be downloaded from the Renfert Website, www.renfert.com, under "Consulting / Repair Service".

12. Standard Delivery

- 1 *Vario basic*
- 1 Foot switch
- 1 Connector kit
- 1 Fluorescent lamp with lamp cover
- 1 Accessories kit
- 1 Pair of sandblasting gloves
- 1 Operating instructions, with attachments
- 1 Spare parts list

13. Delivery Versions

- No. 2960-0000 *Vario basic*, basic unit, 230 V, 50-60 Hz, without sandblasting tanks
- No. 2960-1000 *Vario basic*, basic unit, 120 V, 50-60 Hz without sandblasting tanks

14. Accessories

Micro sandblasting tanks

- No. 2954-0050 25-70 µm, Right / blue
- No. 2954-0051 25-70 µm, Left / red
- No. 2954-0250 70-250 µm, Right / blue
- No. 2954-0251 70-250 µm, Left / red

Wand mounting

- No. 2960-0001 Wall mounting, incl. 90° extractor bracket

Extractor bracket

- No. 2960-0002 90° extractor bracket

Protective grate

- No. 2960-0003 Protective grate for the viewing pane

Blasting nozzle

- No. 90003-5520 Borocarbide nozzle

Abrasive

Cobra aluminium oxide

- No. 1594-1105 25 µm [500 mesh], white 5 kg canister
- No. 1594-1205 50 µm [270 mesh], white 5 kg canister
- No. 1594-2220 50 µm [270 mesh], white 20 kg bucket
- No. 1584-1005 90 µm [170 mesh], white 5 kg canister
- No. 1583-1005 110 µm [150 mesh], white 5 kg canister
- No. 1583-1020 110 µm [150 mesh], white 20 kg bucket
- No. 1587-1005 125 µm [115 mesh], pink 5 kg canister
- No. 1587-1020 125 µm [115 mesh], pink 20 kg bucket
- No. 1585-1005 250 µm [60 mesh], white 5 kg canister
- No. 1585-1020 250 µm [60 mesh], white 20 kg bucket

Rolloblast beads

- No. 1594-1305 50 µm [400-200 mesh] 5 kg canister
- No. 1594-2312 50 µm [400-200 mesh] 12.5 kg canister
- No. 1589-1005 100 µm [170-100 mesh] 5 kg canister

15. Troubleshooting

Problem	Possible cause	Corrective action
No air and no abrasive	<ul style="list-style-type: none"> Nozzle blocked. Nozzle blocked at grip and/or blasting hose blocked. Kink(s) in the foot switch connector hose. Switch set to intermediate position. Pressure set too low. 	<ul style="list-style-type: none"> Open the blasting nozzle unit and clean the nozzle (Figure 26, 27, 28). Unscrew the nozzle from the handle and blow it out with compressed air. Without reconnecting the nozzle, activate the foot switch to blow compressed air through the hose. Inspect and correct the connecting hoses. Turn the switch until you feel it click into place. Open the pressure regulator further (recommended pressure: 4-5 bar).
Air only and no or very little abrasive	<ul style="list-style-type: none"> Dirty or moist abrasive. Insufficient abrasive in the unit. Intake hose blocked. Problems with the additional tank. 	<ul style="list-style-type: none"> Replace the abrasive. Add abrasive. Check that the intake hose is not kinked / pinched / blocked. The intake hose must be securely attached to the unit base (Figure 7). Refer to "Instructions for Installing a Micro Sandblasting Tank on the Vario basic", No. 21-9774
Reduced blasting force	<ul style="list-style-type: none"> Function selector switch not properly set. Nozzle worn. Abrasive used up. Abrasive feed hose is blocked / damaged / kinked. Insufficient abrasive in the reservoir. 	<ul style="list-style-type: none"> Make sure the selector switch clicks into place. Replace the nozzle. Replace the abrasive. Drain the abrasive out and inspect / replace the feed hose. Add abrasive.
Excessive abrasive consumption	<ul style="list-style-type: none"> Extraction power set too high. 	<ul style="list-style-type: none"> Reduce the extraction power.
Blasting chamber fills with dust.	<ul style="list-style-type: none"> Insufficient extraction power. Extractor dust bag full. 	<ul style="list-style-type: none"> Increase extraction pressure by sealing the openings on the back of the unit (e.g., with a piece of tape) (Figure 17 (b)). Clean the extractor / replace the dust bag.
Abrasive spurts in pulses.	<ul style="list-style-type: none"> Nozzle connectors not vertical. Abrasive used up or too dirty. Kink in the suction hose. 	<ul style="list-style-type: none"> Make sure the nozzle is vertical. Replace the abrasive. Inspect the feed hose.
Unit continuously blasting and stops when foot switch is activated.	<ul style="list-style-type: none"> Foot switch connected incorrectly. 	<ul style="list-style-type: none"> Inspect and correct the foot switch connection.
Foot switch fails to switch on.	<ul style="list-style-type: none"> No pressure at the compressed air connection. Kink(s) in the pneumatic hoses. 	<ul style="list-style-type: none"> Inspect the compressed air lines and compressor. Realign the hoses.
Air leaking from the foot switch.	<ul style="list-style-type: none"> Pneumatic hoses connected incorrectly. 	<ul style="list-style-type: none"> Replace the connectors.
Lamp fails to go on.	<ul style="list-style-type: none"> Power cord not connected. Faulty fluorescent lamp. Faulty electronics. 	<ul style="list-style-type: none"> Plug the power cord into the wall outlet. Replace the fluorescent lamp (refer to Section 7.7). Have the Service department replace the electronics or refer to the spare parts list.

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Renfert GmbH, Industriegebiet, 78247 Hilzingen / Germany