

making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - Polski
- FAQ - Čeština
- FAQ - 中文
- FAQ - 日本語
- FAQ - 한국어
- FAQ - اللغة العربية
- FAQ - Українська
- FAQ - Suomi
- FAQ - Svenska

# SILENT TS

FAQ - frequently asked questions

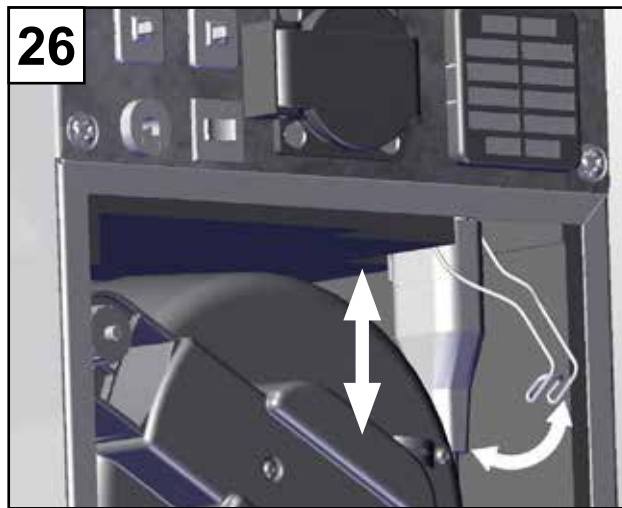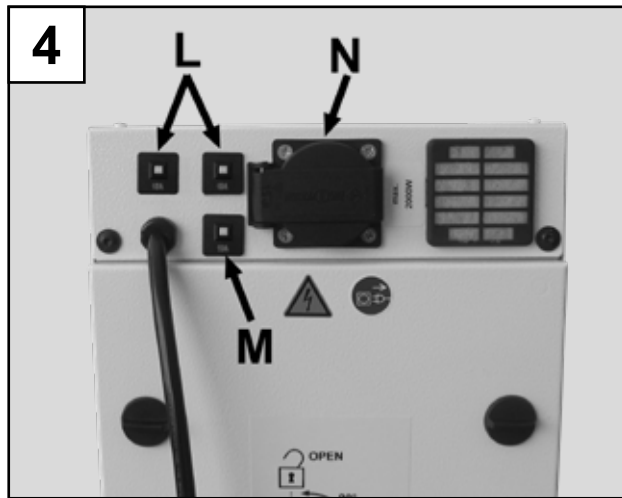*Fig. 1*

## FAQ - Deutsch

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## FAQ - English

**i** For information on operating elements, see [Fig. 1](#) on page 2 of this document.

**i** For chapter indications, refer to the complete instruction manual on the internet.

Error	Possible cause	Corrective action
<b>Dust extractor stops suddenly, for no apparent reason.</b>	<ul style="list-style-type: none"> <li>• Dust extractor continued to be used, despite “Change Filter” indicator.</li> <li>• Unit fuse blown.</li> <li>• Motor protection switch (<a href="#">M. Fig. 4</a>) has been activated due to overload (temperature, electricity).</li> <li>• Motor over-temperature protector switch activated.</li> <li>• Plug on grey motor wire out of its socket or loose.</li> <li>• Suction turbine fault.</li> <li>• In-house fuse blown.</li> </ul>	<ul style="list-style-type: none"> <li>• Check and replace the dust bag and fine particle filter (Chapter 3.2 and 3.3).</li> <li>• Check and reinstall the fuse.</li> <li>• Check the motor protection switch, if necessary reset.</li> <li>• Change the motor if repeatedly activated.</li> <li>• Check dust bag and filter, replace as required. (Sec. 3.2 and 3.3).</li> <li>• Check exhaust air filter, replace as required. (Sec. 3.3.2).</li> <li>• Check whether suction hose is blocked. Remove obstruction as required.</li> <li>• Dust extractor can be restarted after being allowed to cool off for approx. 1 hour.</li> <li>• Check plug (<a href="#">Fig. 26</a>).</li> <li>• Carry out self-diagnosis (chapt. 3.5), at fault „E2“ exchange suction turbine (see spare parts list).</li> <li>• Replace the in-house fuse.</li> <li>• Check the total power consumption of the dust extractor plus dust-generating devices.</li> </ul>
<b>Dust extractor stops suddenly, for no apparent reason. E4 is indicated on the display.</b>	<ul style="list-style-type: none"> <li>• The electronics may be overheated due to excessively high ambient temperature, e.g., if the unit is installed in a cabinet.</li> </ul>	<ul style="list-style-type: none"> <li>• Allow the extractor to cool down.</li> <li>• Attempt to improve the ventilation conditions. Refer also to Sec. 1.1.</li> </ul>
<b>After switching on the unit, the display immediately indicates E4.</b>	<ul style="list-style-type: none"> <li>• Temperature sensor fault.</li> <li>• After overheating, the device has not yet cooled down sufficiently.</li> </ul>	<ul style="list-style-type: none"> <li>• Have the device repaired.</li> <li>• Disconnect the device and let it continue to cool down.</li> </ul>
<b>The self-test returns a suction turbine error, even though the turbine is running.</b>	<ul style="list-style-type: none"> <li>• Filter full.</li> <li>• Suction hose blocked.</li> <li>• Suction turbine fault.</li> </ul>	<ul style="list-style-type: none"> <li>• Check dust bag and fine particle filter, replace as required. (Sec. 3.2 and 3.3.1).</li> <li>• Remove obstruction from the suction hose.</li> <li>• Replace the suction turbine (Sec.3.6).</li> </ul>
<b>Insufficient extraction force.</b>	<ul style="list-style-type: none"> <li>• Extraction force level too low.</li> <li>• Filter blocked.</li> <li>• Suction hose blocked.</li> <li>• Dust drawer seal not tight.</li> <li>• Upper front panel fails to close properly.</li> <li>• Suction hose damaged.</li> </ul>	<ul style="list-style-type: none"> <li>• Increase the extraction level (Sec. 2.3).</li> <li>• Check dust bag and fine particle filter, replace as required.(Sec. 3.2 and 3.3.1).</li> <li>• Remove obstruction from the suction hose.</li> <li>• Check the proper seat of the dust drawer (Sec. 3.2).</li> <li>• Check the dust drawer seal for damage and replace as required (Sec. 3.1).</li> <li>• Check the proper seat of the upper front panel (Sec. 3.3.1).</li> <li>• Check the upper front panel seal for damage and replace as required (Sec. 3.1).</li> <li>• Inspect the suction hose and replace as required.</li> </ul>

Error	Possible cause	Corrective action
<b>“Change Filter” indicator goes on shortly after filters were replaced.</b>	<ul style="list-style-type: none"> <li>• Fine particle filter blocked.</li> <li>• Suction hose blocked.</li> <li>• Cross-section of the dust-generating device’s hose fitting is too small.</li> <li>• Dust is too fine.</li> <li>• Flow sensor defect.</li> </ul>	<ul style="list-style-type: none"> <li>• Replace the fine particle filter (Sec. 3.3.1).</li> <li>• Remove obstruction from the suction hose.</li> <li>• Adjust the sensitivity of the „Replace filter“ indicator(Chapter 2.7.1.).</li> <li>• Adjust the sensitivity of the „Replace filter“ indicator (Chapter 2.7.1.)</li> <li>• Carry out self-diagnosis (chapt. 3.5), at fault „E1“ equipment requires professional repair.</li> </ul>
<b>Dust bag ruptures.</b>	<ul style="list-style-type: none"> <li>• The “Change Filter” indicator’s sensitivity has not been adjusted for the material being extracted.</li> </ul>	<ul style="list-style-type: none"> <li>• Reduce the sensitivity of the „Replace filter“ indicator (Chapter 2.7.1.).</li> </ul>
<b>Dust extractor fails to start when a dust-generating device is being used.</b>	<ul style="list-style-type: none"> <li>• Dust extractor not switched on.</li> <li>• Dust-generating device not plugged into the unit power outlet on the SILENT TS.</li> <li>• Wrong operating mode selected.</li> <li>• Automatic start feature’s threshold value too high.</li> <li>• Device fuse has blown.</li> <li>• Automatic start feature fails to detect the attached device’s operation.</li> </ul>	<ul style="list-style-type: none"> <li>• Switch the dust extractor on (Sec. 2.2).</li> <li>• Connect the dust-generating device to the SILENT TS (Sec. 1.4).</li> <li>• Select the automatic mode (Sec. 2.5).</li> <li>• Adjust the automatic start feature (Sec. 2.6).</li> <li>• Check and reinstall the fuse.(Sec. 3.4).</li> <li>• Some older handheld instruments do not generate a sufficiently strong signal for the automatic start feature. Switch the dust extractor to the continuous mode.</li> </ul>
<b>Dust extractor starts, although a connected device is not being used.</b>	<ul style="list-style-type: none"> <li>• Automatic start feature’s threshold value too low.</li> </ul>	<ul style="list-style-type: none"> <li>• Adjust the automatic start feature (Sec. 2.6).</li> </ul>
<b>Dust extractor fails to stop when a connected device is switched off.</b>	<ul style="list-style-type: none"> <li>• Dust-generating device not plugged into the unit power outlet on the SILENT TS.</li> <li>• Wrong operating mode selected.</li> <li>• Activation threshold too low.</li> </ul>	<ul style="list-style-type: none"> <li>• Connect the dust-generating device to the SILENT TS (Sec. 1.4).</li> <li>• Select the automatic mode (Sec. 2.5).</li> <li>• Adjust the automatic start feature (Sec. 2.6).</li> </ul>
<b>No satisfactory setting of the automatic start feature can be adjusted.</b>	<ul style="list-style-type: none"> <li>• Some older handheld instruments do not generate a sufficiently strong signal for the automatic start feature.</li> </ul>	<ul style="list-style-type: none"> <li>• Switch the dust extractor to the continuous mode.</li> </ul>
<b>Extraction briefly starts up when the unit is switched on.</b>	<ul style="list-style-type: none"> <li>• Activation sensitivity too low.</li> </ul>	<ul style="list-style-type: none"> <li>• Adjust the automatic start feature (Sec. 2.6).</li> </ul>
<b>Panels covering the fine particle filter and the dust drawer are difficult to open.</b>	<ul style="list-style-type: none"> <li>• Lock bolts dirty.</li> </ul>	<ul style="list-style-type: none"> <li>• Clean the lock bolts and lubricate them slightly, if required.</li> </ul>

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