making work easy

de

en

# SIMPLEX model designer

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Made in Germany

21-6053 12042024

making work easy

# SIMPLEX model designer

# Inhalt

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# 1 Introduction

## 1.1 Symbols used

Symbols with the following meanings are used in these instructions or on the equipment:

### Caution

**Risk of damage to the equipment if the instruction is not followed.**

### Note

**Provides the operator with useful information that eases use.**

► List, special attention should be paid to

- List
- Subordinate list

⇒ Instructions / required action / input / operational sequence:

You are asked to carry out the specified action in the order indicated.

◆ Result of an action / response by the equipment / response by the program:

The equipment or program responds as a result of your actions or when a specific incident occurs.

Other symbols are explained as they occur.

## 2 Safety

### 2.1 *Intended use*

Model designer is a CAD software for creating digital working models and planning models from intraoral scan (IOS) data for orthodontic use.

Extending the scope of use beyond the above-mentioned indications is outside the intended use of this software and is therefore not permitted.

## 3 Installation

To download the software, you must log in to „myRenfert“ and register your SIMPLEX model designer.

The link to a Quick Start Guide on how to log in to myRenfert and register the program can be found on the packaging of the copy protection plug (dongle).

After downloading, you must install the software on your computer:

⇒ Open the download directory on your PC.

⇒ Unzip the ZIP file.

⇒ Start the program „modeldesigner\_.....\_installer.exe“ and follow the instructions.

◆ The SIMPELX model designer program will be installed on your PC.

### 3.1 Unlocking the program

The program is protected with a copy protection plug (dongle).

It can only be used if the dongle is connected to the PC:

⇒ Insert the dongle into a free USB port on your PC.

# 4 User interface

When you start the program, you will see the following user interface

- 1 Menu bar
- 2 Toolbars (can be enabled/disabled)
- 3 Action menu, indicates which additional actions are possible with the mouse and keyboard in the current work step.
- 4 Status bar (can be enabled/disabled)

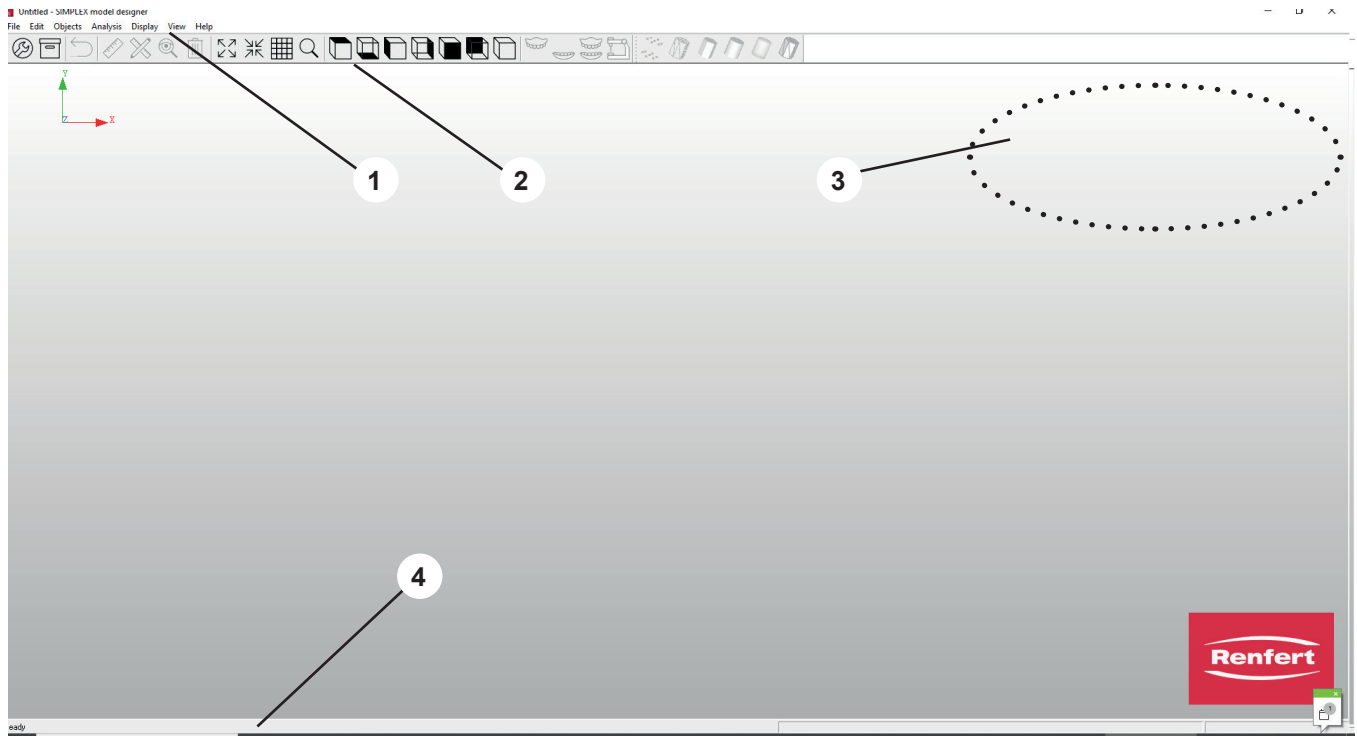
Abb. 1

The menu bar contains all program functions.

For easier and faster access, various functions can also be selected using the icons in the toolbar.

Some functions can also be selected using shortcuts.

### Some program functions are not available!

For the intended use of the program, i. e. creation of planning and working models from intraoral scans, several program functions are not required – therefore they cannot be called up. Although they are mentioned here for the sake of completeness, they are disabled in the software. In these operating instructions they are indicated in “gray” or are marked accordingly.

## 4.1 Toolbars

### ► Functions

Abb. 2

### ► Rendering (Representation)

Abb. 3

# 5 Program functions / operation

## 5.1 Assignment of mouse buttons

- Select / highlight      Left mouse button
- Rotate                    Hold down the right mouse button and move the mouse
- Move/drag                Press mouse wheel and move mouse
- Zoom Rotate the mouse wheel

While editing data, the currently available mouse/keyboard interactions are displayed in a status bar in the top right corner of the screen.

It is also possible to use the 3D mouse (3Dconnexion Spacemouse) for operating the software.

## 5.2 File menu

Icon	Shortcut	Menu item	Description
		New order	Creates a new order and starts the wizard.
		Open order	Opens the database with all created orders.
		Import geometries *)	This function permits to load implemented standard data such as fixation pins.
		Print preview	Displays the print preview.
	Ctrl+P	Print	Prints the current view.
		Print setup	Changes the printer and the print options.
		Exit	Exits the program

\*) Not available in this software release.

## 5.3 Edit menu

Icon	Shortcut	Menu item	Description
	Ctrl+Z	Undo	Undoes the last step.
		Delete objects	Deletes one or more selected objects.
	SHIFT+Delete	Clear all objects	Deletes all loaded objects.

## 5.4 Object menu

Icon	Shortcut	Menu item	Description
		Object info	Displays information about the order and any existing objects.
		Object color *)	Opens a color selection dialog to change the color of an object.
		Mosaic	Displays each object in a different color
		Points	Various display modes for wireframes.
		Wireframe	
		Flat-shaded	
		Gouraud-shaded	
		Transparent	
		Flat + wireframe	
		Show objects *)	Shows selected objects again
		Hide objects *)	Hides selected objects
	S	Toggle "Show/Hide" *)	Toggles between visible and hidden objects
	W	Toggle "Wax layer"	Show or hide the selected item.
	P	Toggle "Occlusal plane"	Show or hide the selected item.
	R	Toggle "Median plane"	Show or hide the selected item.
	B	Toggle "Tuber plane"	Show or hide the selected item.
	S	Toggle "Socket" (i. e. model base)	Show or hide the selected item.
	G	Toggle "Gingival mask"	Show or hide the selected item.
	Z	Toggle "Section plane"	Show or hide the selected item.
	Y	Toggle "Fixation"	Show or hide the selected item.
	T	Toggle "Articulator" *)	Show or hide the selected item.
	A	Toggle "Annotation"	Show or hide the selected item.
	U	Toggle "Upper jaw"	Show or hide the selected item.
	L	Toggle "Lower jaw"	Show or hide the selected item.
	O	Toggle "Pre-op scan"	Show or hide the selected item.
	X	Toggle "Upper / lower jaw"	Changes the view.

\*) Not available in this software release.

## 5.5 Analysis menu

Icon	Shortcut	Menu item	Description
		Point distance 3D	Starts "Point Distance 3D", a function that allows you to select points on the user interface by left-clicking the mouse and to display their 3D distance to each other.
		Point distance 2D	Starts "Point Distance 2D", a function that allows you to select points on the user interface by left-clicking the mouse and to display their 2D distance to each other. As you move the mouse, the 2D distance between this point and the current mouse position is displayed in real time.
	Ctrl+O	Toggle "Occlusion plot"	Show or hide the selected item.
		Material thickness	Another useful tool for analyzing the created model is to output the material thickness. The different material thicknesses are indicated in different colors. Move the mouse over the user interface to display the exact values at the respective points.

## 5.6 Display menu

Icon	Shortcut	Menu item	Description
		Rotation X-axis Y-axis Z-axis Free	Sets the rotation parameters based on individual axes or without any restrictions
		Predefined views	Sets the view to one of seven predefined perspectives.
	Ctrl+F2	Top	
	Ctrl+F3	Bottom	
	Ctrl+F4	Left	
	Ctrl+F5	Right	
	Ctrl+F6	Front	
	Ctrl+F7	Back	
	Ctrl+F8	Isometric	
		Reset view	Resets the 3D Viewer to default (centered from top).
	F6	Center view	Display models in the center of the screen.
		Ignore colors	Ignores the automatic detection of color markers when defining the borders.
	Ctrl+G	Draw grid	Show or hide the selected item.
		Set rotation point	Sets the rotation point on the 3D model for future view rotations.
		Rectangular zoom	Set the zoom range by moving the mouse while holding down the left mouse button.
		Store Viewport	Saves the current view parameters for later use
		Restore Viewport	Sets the view according to the previously saved parameters
	Ctrl+C	Snap screen	Copies screenshot to clipboard

## 5.7 View menu

Icon	Shortcut	Menu item	Description
		Toggle "Toolbars"	Show or hide the selected item.
		Toggle "Status bar"	Show or hide the selected item.
		Customizing	Customer-specific adaptations can be loaded here, e. g. additional desired geometry shapes / adapters.
		Model types	Default values for designing a model type can be entered here.
		Options	Displays general settings / options.

### 5.7.1 Model types

This feature is currently not yet available.

### 5.7.2 Options

**Caution**  
Changes made here will not take effect until the program is restarted.

"Current folder" indicates the path to the database.

This can be changed via "New folder".

"Import folder" specifies the path for importing scan data. Can be set from the drop-down list.

"Export folder" specifies the path to an export folder for the design STL. Can be set from the drop-down list.

"Language" specifies the wizard language.

"Import reduction" reduces the scan data based on the selected accuracy value.

"Export reduction" reduces the model data based on the selected accuracy value.

"Automatic saving" – if unchecked, the data will not be saved temporarily.

"Basal chamfer" chamfers the dental arch basally.

"Occlusion plane" defines the jaw data set on which the occlusion plane is created.

"Result rotation": Adjustable orientation of the design after clicking the "Export Model" button in the "Finalize" area for correct transmission to the 3D printer.

Abb. 4

## 6 Planning model – design process

A wizard guides the user through the process of creating a planning model in 5 steps.

It is only possible to move to the next step after completing the previous step with all required entries and actions.

Before that, the “Next” button is disabled.

A planning model is created in the following 5 steps:

1. Order
2. Import
3. Direction
4. Basis (Model base)
5. Finalize

The wizard is started via “File – New order” or “File – Open order”

⇒ File – New order:

- ◆ A new order is created.

⇒ File – Open order:

- ◆ This function permits to continue editing an existing order. A selection window will open showing the orders that have already been created. Select the desired order and load it with “OK”.

**A reloaded order will always be opened in the step in which it was closed in the previous session.**

**To make changes again, you can go back to the previous step from any step using the “Back” button.**

### 6.1 Order (step 1/5)

Enter the patient data and select whether to create a working or a planning model.

- 1 Job ID: Sequential number assigned by the system.
- 2 Patient: The patient's name (optional).
- 3 Type: Selection of previously created model types including their parameters, see also chapter 5.7.1. \*)
- 4 Technician: Name of the technician (optional).
- 5 Date: Assigned by the system, cannot be changed.
- 6 Comment: Enter additional, individual details about the order (optional).
- 7 Task: Select whether to create a planning model or a working model. Select “Planning model”!
- 8 Save your entries
- 9 Finalize this step and open the next step.

\*) Not available in this software release.

Abb. 5

## 6.2 Import (step 2/5)

Load the scan data. You can load the following file formats:

- STL
- PLY with color information

- 1 Import upper jaw
- 2 Import lower jaw
- 3 Import pre-op scan (optional)
- 4 Trim scan (refer to 6.2.1)
- 5 Show penetration (refer to 6.2.2)
- 6 Save your entries
- 7 Finalize this step and open the next step.

To create a planning model, an upper and a lower jaw must always be loaded. You can only proceed to the next step by clicking the "Next" button after both scans have been loaded.

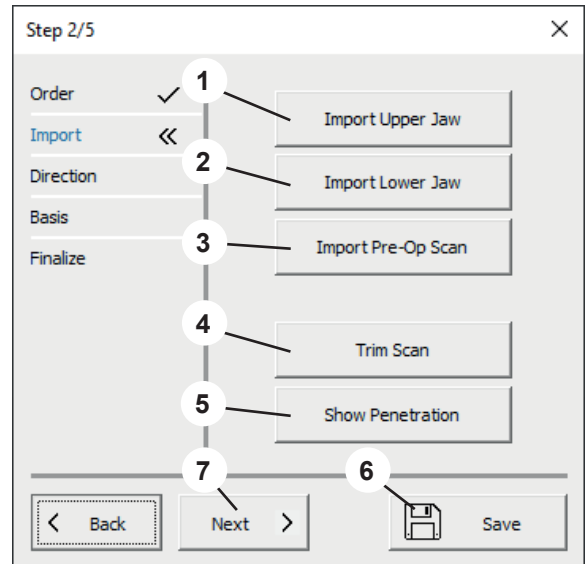
Abb. 6

### 6.2.1 Trim scan

- 1 A field can be drawn by clicking with the left mouse button. Double-click with the left mouse button to delete the data within the field.
- 2 The possible mouse and keyboard actions are displayed in an action field.

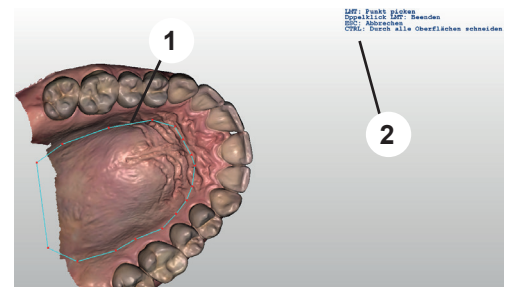
Abb. 7

### 6.2.2 Show penetration

This view shows the premature contacts in static occlusion.

- ⇒ Press the Enter key.
  - ◆ A PDF file is created as a penetration protocol.
- ⇒ Press the ESC key.
  - ◆ The view is closed.

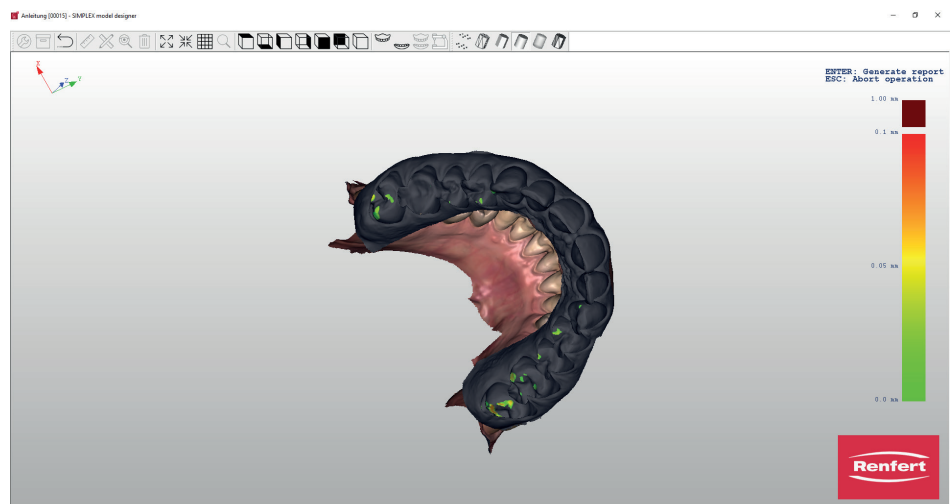
Abb. 8

## 6.3 Direction (step 3/5)

- 1 Define occlusal plane (refer to 6.3.1)
- 2 Define median plane (refer to 6.3.2)
- 3 Define tuber plane (refer to 6.3.3)
- 4 Save your entries
- 5 Finalize this step and open the next step.

The definition of the planes must then be confirmed by pressing the Enter key.

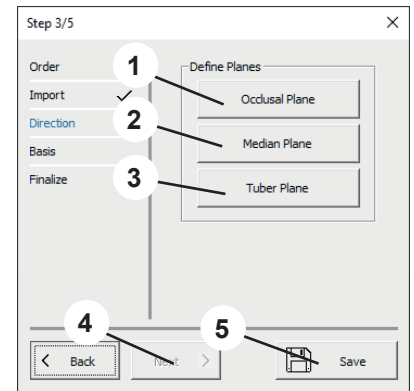
Abb. 9

### 6.3.1 Define occlusal plane

The mouse cursor changes to a crosshair.

The jaw defined in the settings for defining the occlusal plane is displayed.

The three highest points (usually in the middle of the incisors and on the large molars) must be selected by clicking on them with the left mouse button.

The occlusal plane can be selected by clicking on the arrow in the middle (arrow and plane change color) and then tilted by holding down the left mouse button, or moved in height by rotating the mouse wheel.

The three points can also be moved by pressing and holding down the left mouse button. The occlusal plane will change accordingly.

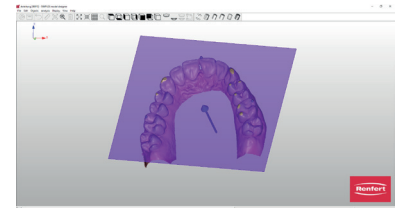
Abb. 10

### 6.3.2 Define median plane

The mouse cursor changes to a crosshair.

Left-click on the first median point according to the model analysis (e.g., papilla incisiva).

Move the mouse to draw a straight line from this point. The median plane is determined by clicking on another reference point (e.g. palatal suture).

The median plane can be precisely adjusted by clicking and dragging the corner points.

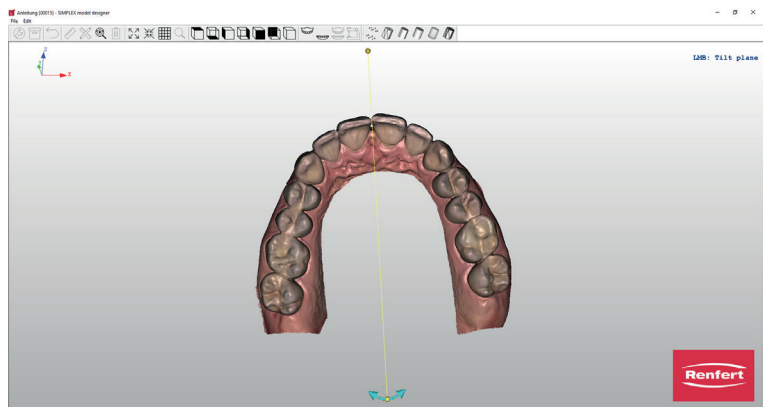
Abb. 11

### 6.3.3 Define tuber plane

The tuber plane is automatically created perpendicular to the occlusal plane and the median plane.

The exact position can be changed by rotating the mouse wheel.

When the tuber plane is in the desired position, press Enter to confirm all planes.

If the occlusal plane is subsequently changed, the median and tuber planes are deleted and must be repositioned. If the median plane is subsequently changed, the tuber plane is deleted and must be recreated.

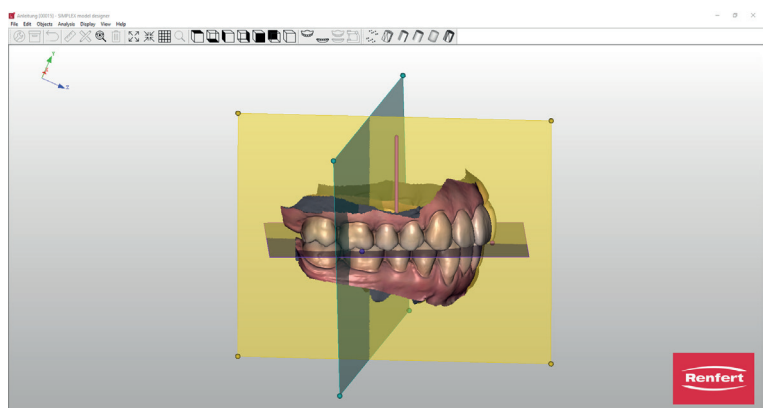
Abb. 12

## 6.4 Basis (Model base) (step 4/5)

- 1 Socket (model base) / Form (Shape): Select and position the desired base shape
- 2 Save your entries
- 3 Finalize this step and open the next step.

Check that the selected base size matches the model.

If the base is too small or if the model is placed outside the base, an error message is displayed.

The selected model bases are displayed on the model.

The position of the model bases can be changed.

⇒ Move the mouse cursor to one of the model bases:

- ◆ Forward/backward arrows appear on screen.
- ◆ Use the mouse wheel to move both model bases forwards / backwards.

⇒ Move the mouse cursor to one of the red control points of the forward/back arrows:

- ◆ The arrow changes to an up/down arrow
- ◆ This jaw can be moved up/down with the mouse wheel.

The base types "...variable" can be scaled by pressing the mouse button and Control key.

The yellow line on the jaw model defines the boundary line for the base shape.

This line can be moved by clicking on individual points and dragging the mouse.

If a point is activated with the mouse and the mouse wheel is rotated, all points move up/down.

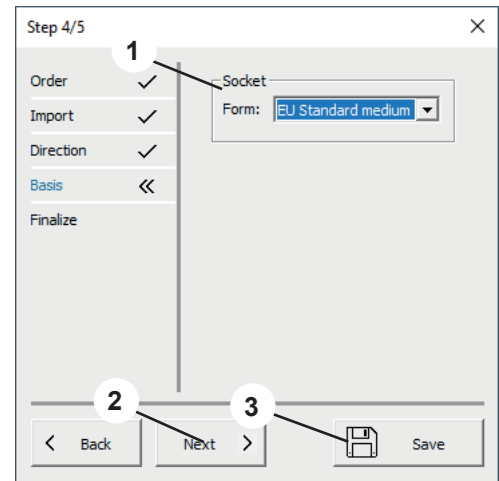
Abb. 13

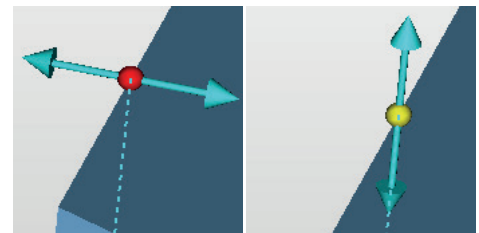
Abb. 14

## 6.5 Finalize (step 5/5)


The models are calculated with the entered parameters.

Various adjustments can now be made.

**The model to which the function is to be applied can be determined by selecting "Upper Jaw / Lower Jaw".**

- 1 Engrave model (refer to 6.5.1)
- 2 Finalize model (refer to 6.5.2)
- 3 Export model (refer to 6.5.3)
- 4 Open sliceware
- 5 Upper jaw / Lower jaw:
- 6 Save your entries
- 7 "Exit" closes the wizard.

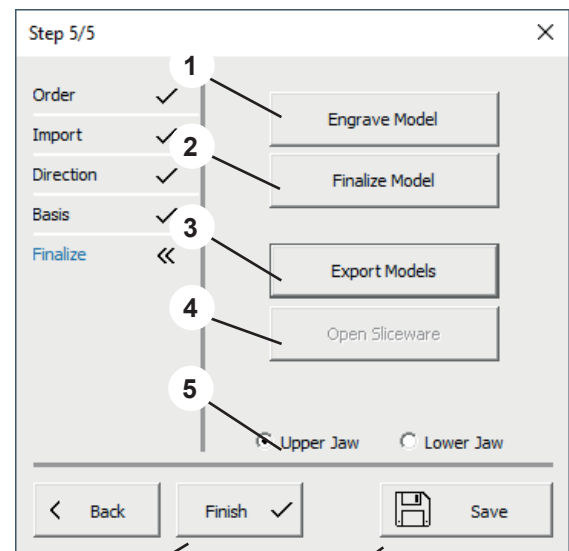7

6

Abb. 15

### 6.5.1 Engrave Model

The mouse cursor changes to a crosshair.

A menu window opens in which the following entries are possible:

- Enter the desired engraving text; the patient's name is automatically suggested.
- Select whether the engraved lettering should be embossed or raised.
- Embossing thickness

Click the left mouse button to define the position of the text.

Left-click and drag the text to the desired position.

Rotate the text by right-clicking and dragging it.

Click "OK" to execute the action.

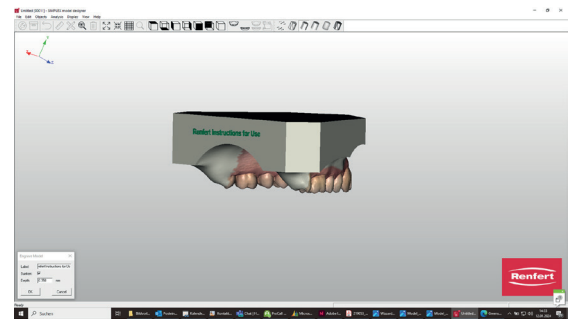
Abb. 16

### 6.5.2 Finalize model

The mouse cursor becomes a selection tool.

The size can be adjusted by holding down the Shift key and rotating the mouse wheel.

The displayed parameters are applied to the selection tool based on the selected action (apply material, remove material, smooth surface).

#### Intensity / Depth

**Intensity:** Can be set via a percentage (%); the function reacts to the duration of the mouse click.

**Thickness:** Can be set during application and removal and allows for uniform application/removal in mm.

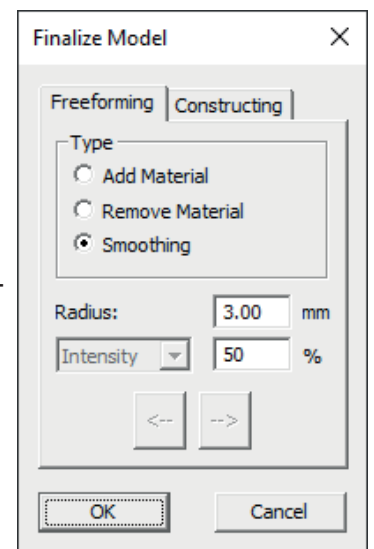
Abb. 17

### 6.5.3 Export models

Click this button to open the Windows Explorer.

If a path has been defined under "Settings – Export folder", it will be opened.

You can also navigate to the desired export folder and save the data records directly there as open STL files.

## 7 Working model – design process

A working model is created in 6 steps:

1. Order
2. Import
3. Direction
4. Block out
5. Socket (Model base)
6. Finalize

The wizard is started via "File – New order" or "File – Open order"

⇒ File – New order:

- ◆ A new order is created.

⇒ File – Open order:

- ◆ This function permits to continue editing an existing order. A selection window will open showing the orders that have already been created. Select the desired order and load it with "OK".

**A reloaded order will always be opened in the step in which it was closed in the previous session.**

The steps "Order" (step 1/6), "Import" (step 2/6) and "Finalize" (step 6/6) are identical to the steps for creating a planning model (see chapters 6.1, 6.2, 6.5) and are therefore not described here.

## 7.1 Order (step 1/6)

Refer to chapter 6.1

When selecting whether a planning model or working model is to be created, select "Working model"!

## 7.2 Import (step 2/6)

Refer to chapter 6.2

## 7.3 Direction (step 3/6)

- 1 Define occlusal plane (refer to 7.3.1)
- 2 Define median plane (refer to 7.3.2)
- 3 Insertion direction (refer to 7.3.3)
- 4 Upper jaw / Lower jaw:
- 5 Save your entries
- 6 Finalize this step and open the next step.

**i** The model to which the function is to be applied can be determined by selecting "Upper Jaw / Lower Jaw".

**i** The definition of the planes must then be confirmed by pressing the Enter key.

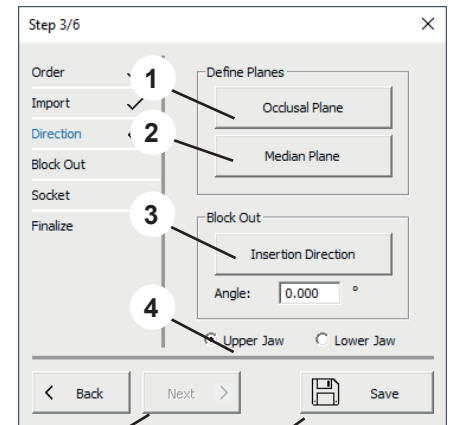
Abb. 18

### 7.3.1 Define occlusal plane

The jaw that was defined under the occlusal plane settings is automatically displayed. The three highest points (usually in the middle of the incisors and on the large molars) must be selected by clicking with the left mouse button on them. The created plane can also be tilted in the viewing direction by selecting it with the mouse (arrow changes color) and left-clicking the mouse; the height can be adjusted by rotating the mouse wheel.

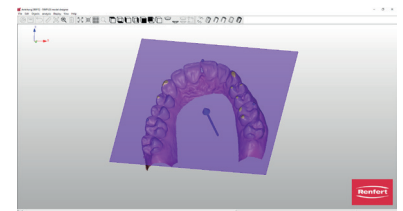
Abb. 19

### 7.3.2 Define median plane

The mouse cursor changes to a cross-hair. The first median point should be clicked with the left mouse button according to the model analysis (e.g., papilla incisiva). Move the mouse to draw a straight line from this point. The median plane is determined by clicking on another reference point (e.g. palatal suture). This can be fine-adjusted by moving the mouse to the end points of the plane while holding down the mouse button.

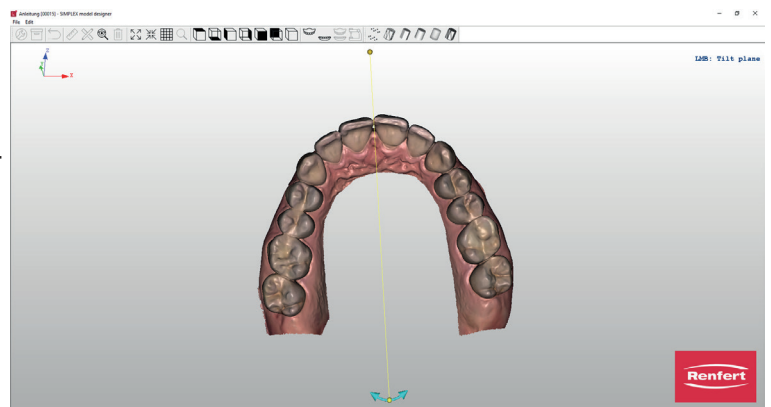
Abb. 20

### 7.3.3 Insertion direction

Blocking out in the insertion direction is possible by clicking on the "Insertion direction" button.

This divides the software window into three different areas. By rotating the model in the large left-hand window (hold down the mouse cursor and left mouse button), the undercuts change according to the viewing direction.

The changed undercut areas are visualized in the small windows on the right.

The views in the right-hand windows can be freely defined using the mouse cursor and holding down the left mouse button without changing the selected insertion direction. If the mouse cursor is moved over undercut areas, a numerical value appears which describes the depth of the undercut at this point.

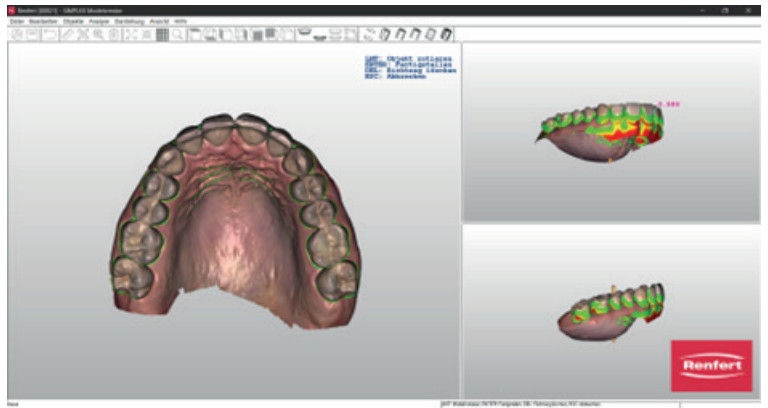
Abb. 21

⇒ Enter key:

- ◆ The newly set insertion direction is accepted.

⇒ ESC key:

- ◆ Cancels the selection.

⇒ DEL key:

- ◆ Deletes the insertion direction already selected.

“Angle” can be used to define an angle for the block-out wax in relation to the insertion direction.


### 7.4 Block out (step 4/6)

If an insertion direction has been entered, all undercut areas are automatically blocked out.

- 1 Alter wax layer (refer to 7.4.1)
- 2 Upper jaw / Lower jaw:
- 3 Save your entries
- 4 Finalize this step and open the next step.

The model to which the function is to be applied can be determined by selecting “Upper Jaw / Lower Jaw”.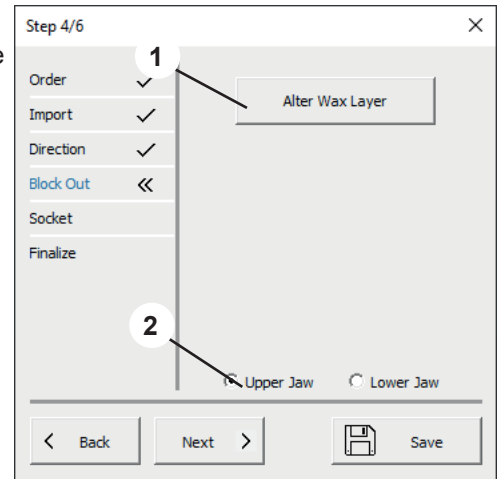

4

3

Abb. 22

## 7.4.1 Alter wax layer

Additional application, removal and smoothing of block-out wax.

The mouse cursor becomes a selection tool.

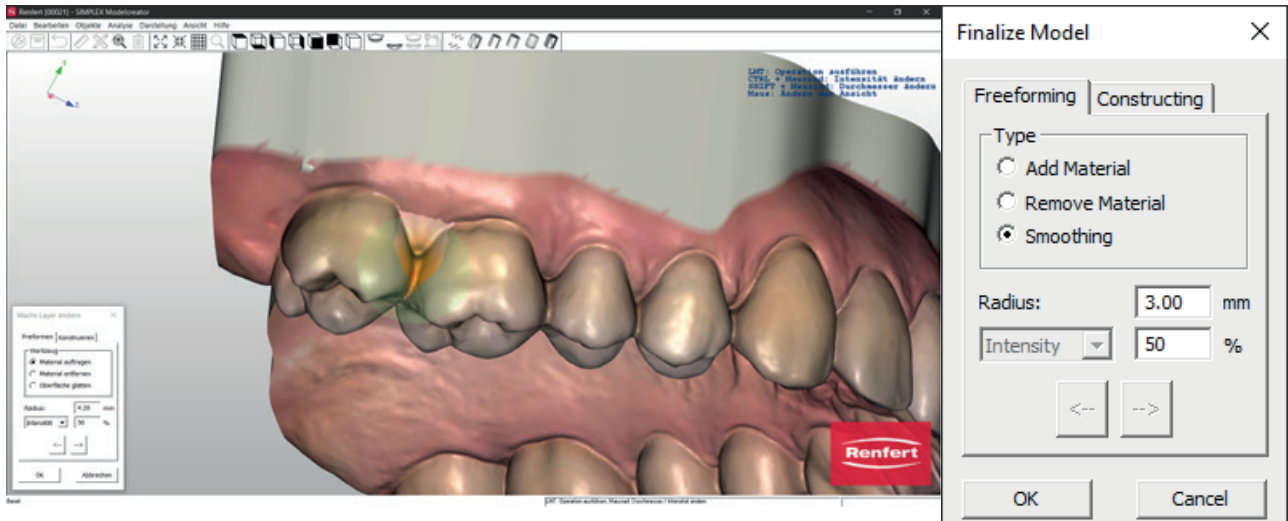
Abb. 23

The size can be adjusted by holding down the Shift key and rotating the mouse wheel.

The displayed parameters are applied to the selection tool based on the selected action (apply material, remove material, smooth surface).

### Intensity / Depth (Thickness)

**Intensity:** Can be set via a percentage (%); the function reacts to the duration of the mouse click.

**Depth (Thickness):** Can be set during application and removal and allows for uniform application/removal in mm.

## 7.5 Model base (Socket; step 5/6)

You can now define base plates and, if required, articulator supports.

- 1 Socket (Model base) (refer to 7.5.1)
- 2 Fixation
- 3 Upper jaw / Lower jaw:
- 4 Save your entries
- 5 Finalize this step and open the next step.

The model to which the function is to be applied can be determined by selecting "Upper Jaw / Lower Jaw".

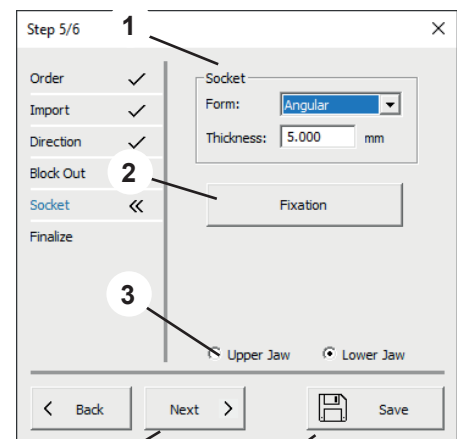
Abb. 24

### 7.5.1 Socket (Model base)

**Form (Shape):**

Various base shapes are available. They can be selected via the drop-down menu.

**Thickness:**

Determines the thickness of the base plate.

If the mouse cursor is moved over the base plate, the color changes and the object is enabled.

If you hold down the left mouse button, you can move the base plate in XY direction.

Rotate the mouse wheel to scale the base plate.

Move the mouse to the end of the base plate to enable a crosshair.

Move the mouse cursor to the corresponding axis and rotate the mouse wheel to scale the base plates separately in XY extension or to adjust the height.

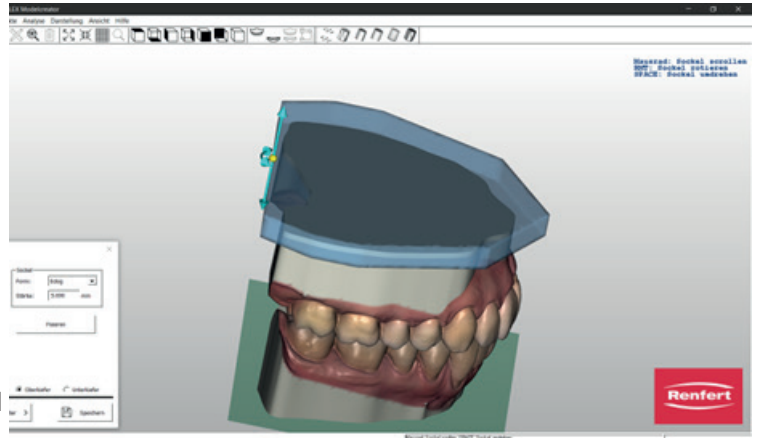
Abb. 25

### 7.6 Finalize (step 6/6)

Refer to chapter 6.5

**Hochaktuell und ausführlich auf ...  
Up to date and in detail at ...  
Actualisé et détaillé sous ...  
Aggiornato e dettagliato su ...  
La máxima actualidad y detalle en ...  
Актуально и подробно на ...**

**[www.renfert.com](http://www.renfert.com)**

Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany  
Tel.: +49 7731 82 08-0 • Fax: +49 7731 82 08-70  
[www.renfert.com](http://www.renfert.com) • [info@renfert.com](mailto:info@renfert.com)

Renfert USA • 3718 Illinois Avenue • St. Charles IL 60174/USA  
Tel.: +1 6307 62 18 03 • Fax: +1 6307 62 97 87  
[www.renfert.com](http://www.renfert.com) • [info@renfertusa.com](mailto:info@renfertusa.com)  
USA: Free call 800 336 7422