
making work easy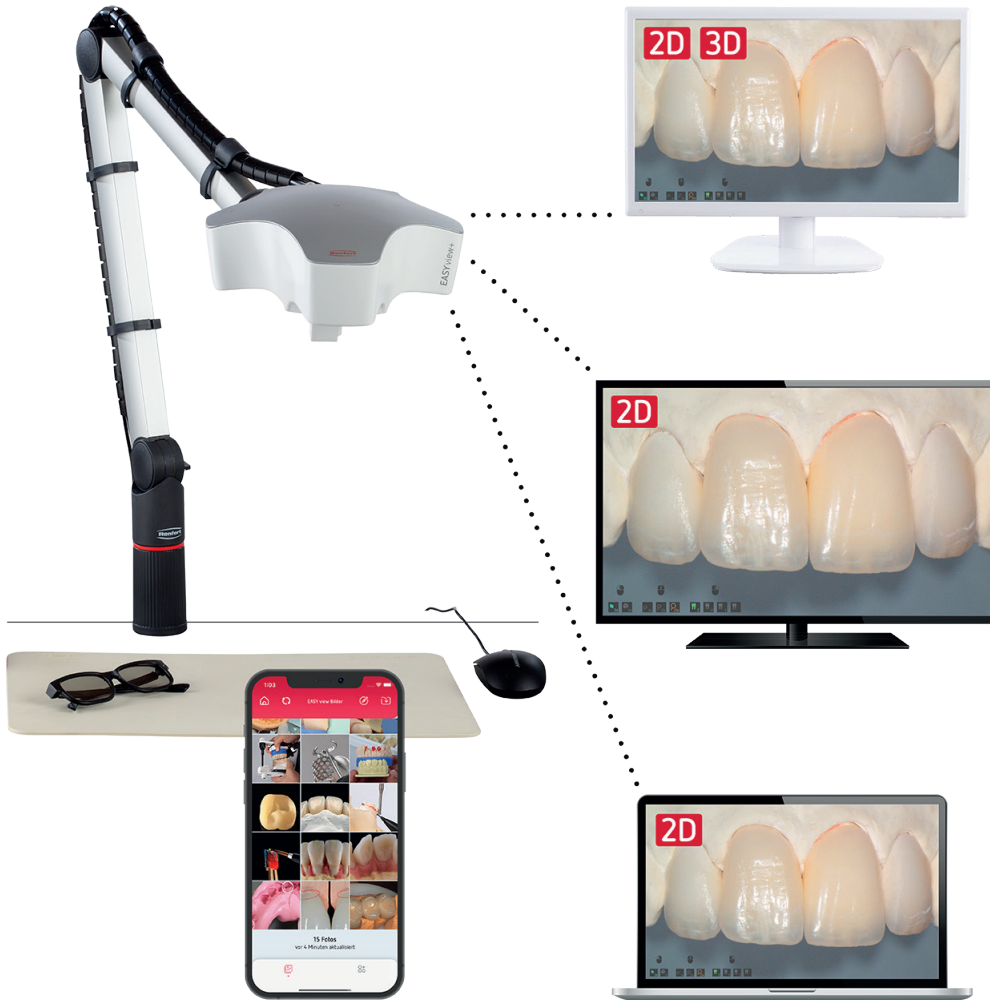

- de
- en
- fr
- it
- es
- pt
- ru

# Software Update

making work easy

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making work easy

# EASY view+ / EASY view+ 3D / EASY view 3D Software update


## Update - options

- EASY view+ or EASY view+ 3D:
- Renfert CONNECT app or
  - [stick \(USB flash drive\)](#)
- EASY view 3D:
- [USB flash drive \(only\)](#)

## Update via Renfert CONNECT app


- i** *If you have the Renfert CONNECT app, a dot on the bell icon indicates that an update is available for one of your units that is connected to the CONNECT app.*
- i** *To perform an update using the Renfert CONNECT app, the EASY view+ must be equipped with the Renfert CONNECT stick (USB flash drive) and the EASY view+ must be registered in the Renfert CONNECT app.*

⇒ Tap the bell icon.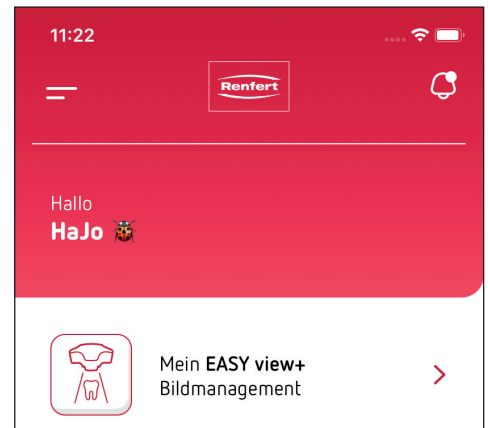


◆ Available updates are displayed.

⇒ Tap "To Install".

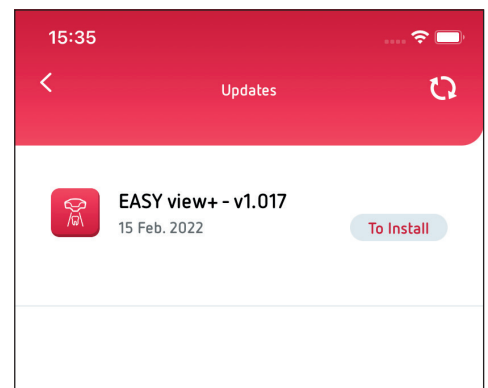
- ◆ Release notes are shown for this update.
- ⇒ Read through the release notes for this update.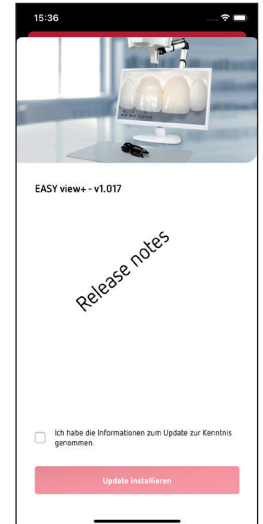


- ⇒ Confirm that you have noted this information.


- ⇒ Tap "Install update".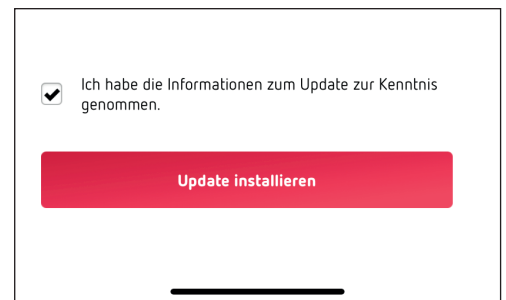

- ◆ The update is transferred to the Renfert CONNECT stick.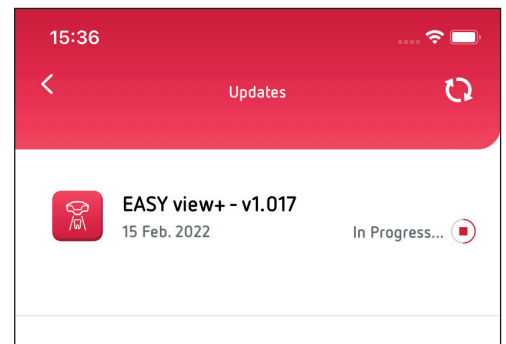

- ◆ The display indicates when the update has been successfully transferred.

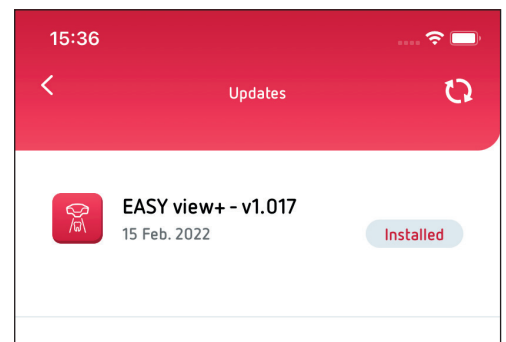
- ◆ The update now runs automatically on the EASY view+, and on the EASY view+ screen, a gray Renfert logo is shown first followed by a red one.
- ◆ The update is complete when the EASY view+ shows a live image again after restarting automatically.
- ◆ The current version number is shown on the bottom right of the monitor.

**CAUTION!**

**Do NOT switch off the EASY view+ until the update has been completed and a live image is shown again!  
Disregarding this information results in irreparable damage to the unit!**

⇒ Once the update has been completed, the live image is shown again, and the new version number is shown on the bottom right of the monitor, the EASY view+ must be switched off then on again.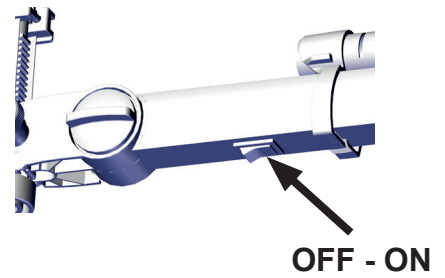

*After switching on again you can work as usual with the EASY view+ and the Renfert CONNECT app.*


# Update using the USB flash drive

en

**i** Using the USB flash drive, you can perform an update on all EASY view units (EASY view 3D, EASY view+ and EASY view+ 3D).

- ⇒ Download the zip file "Vxxx.zip" with the update from the web page [www.renfert.com/...](http://www.renfert.com/...) to a computer.
- ⇒ Open and unpack the ZIP file.  
The unpacked folder contains the instructions, release notes, and the firmware update.


[www.renfert.de/.....](http://www.renfert.de/.....)

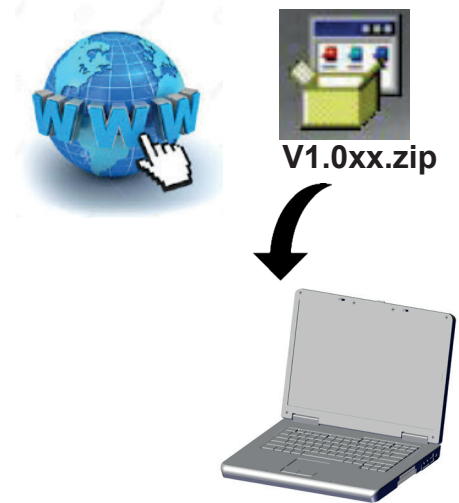- ⇒ Remove the USB flash drive from the EASY view and connect it to the computer.


**Only use the Renfert stick (USB flash drive) provided with the EASY view.  
If another USB flash drive is used, correct installation of the update cannot be guaranteed.**

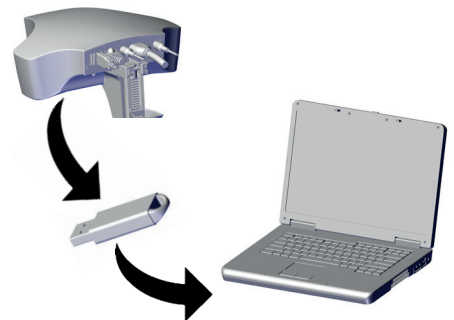
- ⇒ Copy the file with the file extension ".sdfp" to the USB flash drive.

**Do not save the file in a sub-folder.  
Other .sdfp files are not permitted on the USB flash drive.**

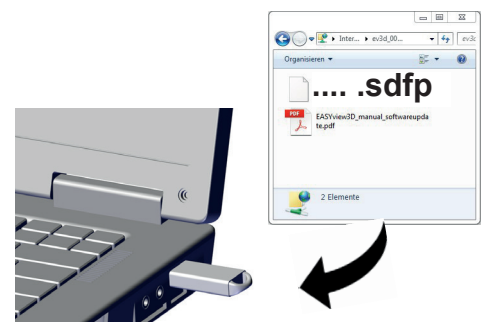
⇒ Connect the USB flash drive to the EASY view.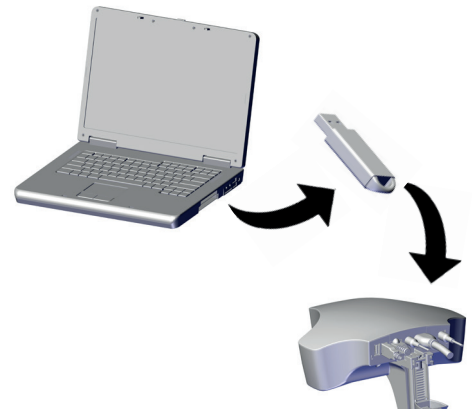

en

⇒ Switch the EASY view off and then on again.

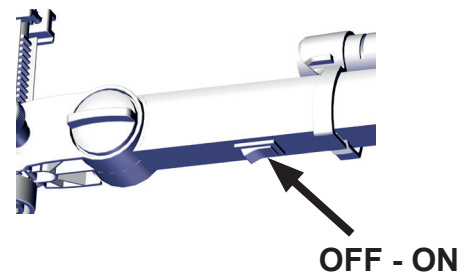- ◆ The update now runs automatically on the EASY view+, and on the EASY view+ screen, a gray Renfert logo is shown first followed by a red one.
- ◆ The update is complete when the EASY view shows a live image again after restarting automatically.
- ◆ The current version number is shown on the bottom right of the monitor.

**CAUTION!**

**Do NOT switch off the EASY view until the update has been completed and a live image is shown again!  
Disregarding this information results in irreparable damage to the unit!**

***As soon as a live image is shown again you can work as usual with the EASY view.***

***If you own an EASY view+ with a Renfert CONNECT stick (USB flash drive) and perform the update as described here using a USB flash drive, you must switch the EASY view+ off and then on again after the update has been completed in order to be able to continue using the Renfert CONNECT app again.***

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