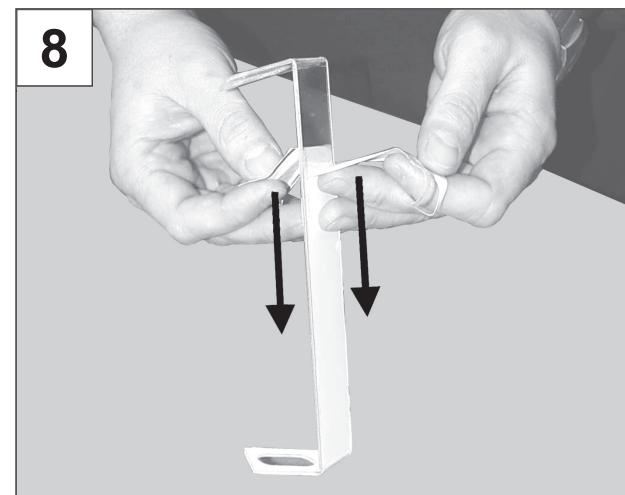
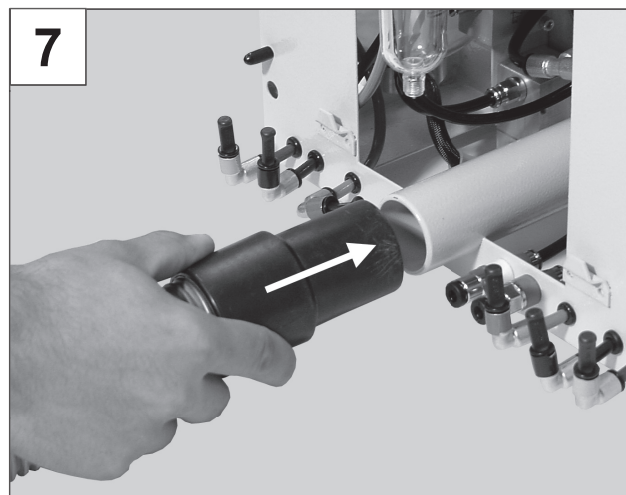
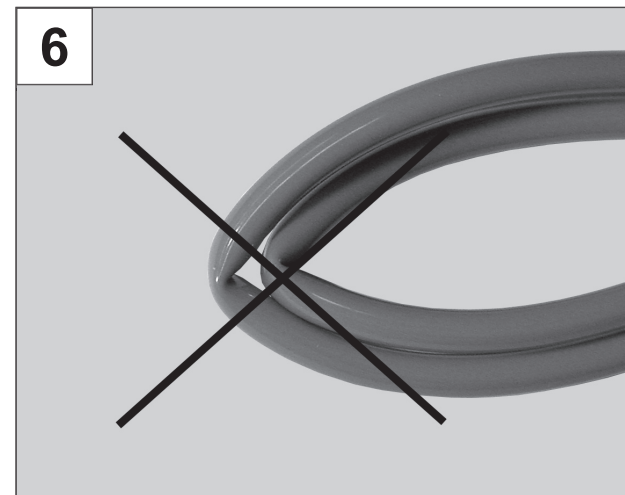
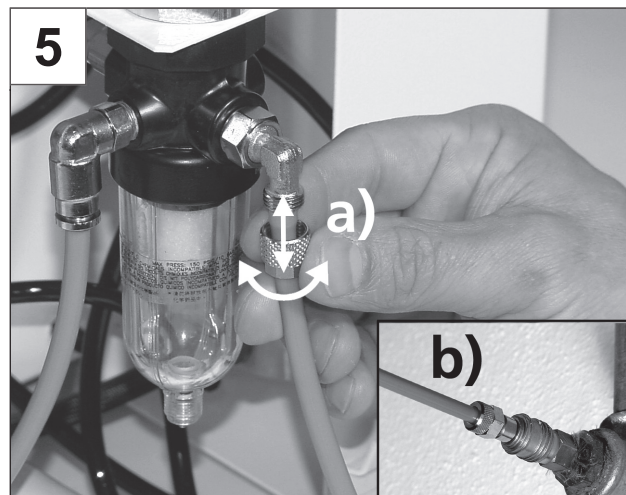
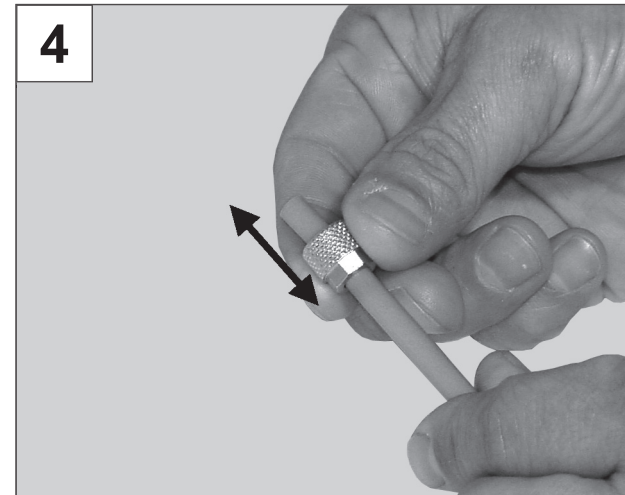
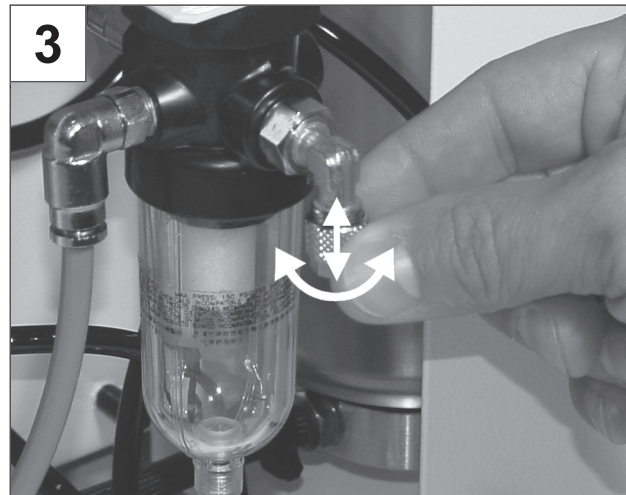
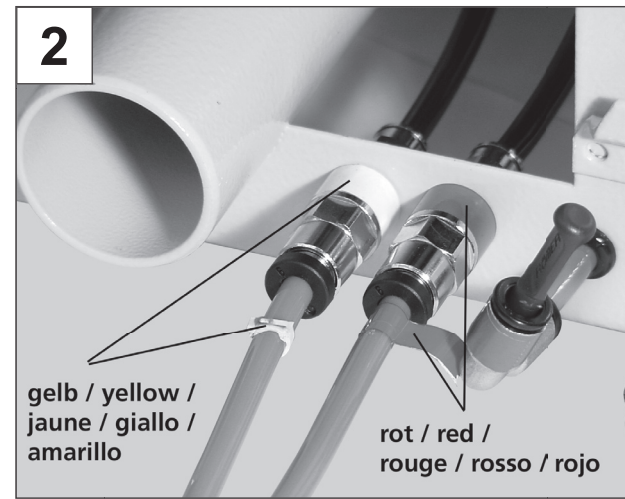
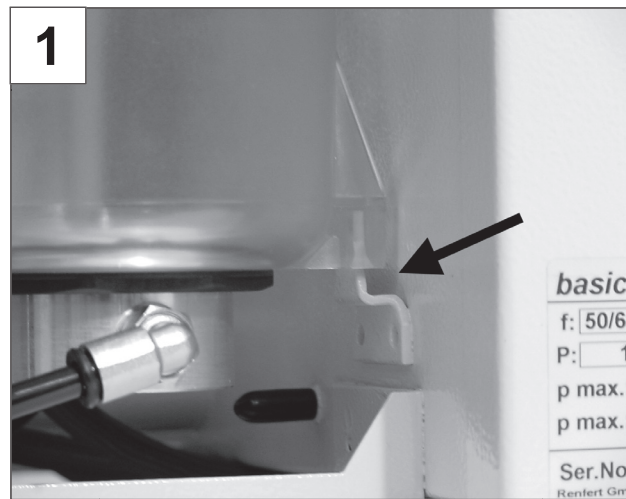
basic quattro IS

No. 2955-0000 / 2955-1000

Bedienungsanleitung
Instruction manual • Mode d'emploi • Istruzioni d'uso
Instrucciones para el servicio
Инструкция по эксплуатации
取扱説明書 • 操作手冊 • 사용 설명서

Seriennummer, Herstelldatum und Geräte-Version befinden sich auf dem Geräte-Typenschild.
 Serial number, date of manufacturing and unit version are shown on the type plate of the unit.
 Le numéro de série, la date de fabrication et la version se trouvent sur la plaque signalétique de l'appareil.
 Il numero di serie, la data di costruzione e la versione si trovano sulla targhetta dell'apparecchio.
 El número de serie, la fecha de fabricación y la versión del aparato están indicados en la placa identificadora del aparato.
 Номер серии, дата изготовления и модель прибора указаны на фирменной табличке.

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ENGLISH

1. Introduction

We are pleased with your decision to purchase the *basic quattro IS*.

This fine sandblasting unit is equipped with an automatic quick-stop (**IS** = Immediate Stop). This allows the abrasive flow to be interrupted within a fraction of a second.

2. Application Area

The *basic* line of sandblasting units are designed for use in dental laboratories to remove investment residues, oxides on cast objects, and for surface treatment. Because of its automatic quick-stop, the *basic quattro IS* is ideally suited for precise, pinpoint application of the abrasive. Special areas of application also include deflasking pressed porcelains or sandblasting porcelain surfaces.

2.1 Ambient Conditions

(in accordance with DIN EN 61010-1)

The *basic quattro IS* may only be operated:

- Indoors;
- Up to an altitude of 2,000 m [6,500 ft] above sea level;
- At an ambient temperature of 5 - 40°C [41 - 104°F] *);
- At a maximum relative humidity of 80% at 31°C [87.8°F], decreasing linearly to 50% relative humidity at 40°C [104°F] *);
- With a mains power supply where current fluctuations do not exceed 10% of the nominal value;
- Under level 2 contamination conditions;
- Under overvoltage category II conditions.

*) Between 5 - 30°C [41 - 86°F], the unit can be operated at a relative humidity of up to 80%. At temperatures between 31 - 40°C [87.8 - 104°F], the humidity must decrease proportionally in order to ensure operational readiness (e.g., at 35°C [95°F] = 65% humidity; at 40°C [104°F] = 50% humidity). The unit may not be operated at temperatures above 40°C [104°F].

3. Safety Information

- Always disconnect the unit from the compressed air and power sources prior to beginning any maintenance work.
- When working on the tank (filling up, cleaning, maintenance) wear protective glasses for protecting your eyes.
- Any residue of abrasive material on the gasket may lead to leakage and early wear of the gasket. After filling up clean the thread and the gasket and close the lid properly.

- Solvents and tensides can create micro-cracking in the plastic (danger of explosion!). Clean the tank and the lid only with a dry cloth. Do not write or stick something on the tanks.
- Check the tanks and the lids regularly on damages and replace them in case of doubt.
- Check the tank cover for correct seating prior to beginning operation. Covers that are not securely closed can suddenly be blown off as the tank pressure increases. The resulting flying parts and abrasive represent a serious hazard.
- **Never** operate the blasting units without suitable dust extraction and appropriate protective gear, as this could result in **health hazards**. The type of dust extraction should be selected on the basis of the dust being generated. Follow here **absolutely** EN 60335-2-69 appendix AA or ask the appropriate authorities.
- Improper use can result in the risk of eye or skin injuries.
- Never direct the blasting material towards eyes or unprotected areas of skin!
- Never work with the view screen open!

Caution: Failure to wear proper eye protection can result in eye injury

- due to particles suspended in the air. Always wear proper eye protection to protect your vision!
- Do not employ the foot switch when only compressed air supply hose is connected. The loose hose could wrap around itself and cause serious injury.
- Unplug the unit from the wall outlet before changing the fluorescent tube.
- Do not squeeze or bend the fluorescent tube body when replacing the tube (breakage hazard). Always wear gloves or use a towel to protect your hands against possible glass splinters.
- Inspect power cords before beginning operation. The unit may not be operated if the power cord is damaged.

3.1 Liability Exclusion

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- The product is employed for any purposes other than those cited in the operating instructions;
- The product is altered in any way other than those alterations described in the operating instructions;

- The product is repaired by other than an authorized facility or if any but Renfert OEM parts are employed;
- The product continues to be employed, despite obvious safety faults.

4. Installation / Commissioning

4.1 Setting up and Connecting the Unit

1. Inspect the tank for proper installation (Fig. 1).
2. Check the voltage information on the nameplate.
3. The unit is operational with two tanks. To connect additional tanks, please refer to Installation Instructions, No. 21-9499.
4. Remove the foot switch from its shipping location in the blasting chamber.
5. Connect the foot switch to the sandblasting unit (red + yellow). Make sure to both pneumatic hoses are fully inserted up to their stops (Fig. 2).
6. Connect the compressed air supply to the sandblasting unit. To do this, loosen the screw-on air coupling on the unit (Fig. 3).
7. Push the screw-on coupling over the supplied blue compressed air hose (Fig. 4).
8. Connect the compressed air hose to the unit and tighten the coupling sleeve (Fig. 5a).
9. Connect the compressed air hose to the compressed air supply (Fig. 5b).

Caution: Minimum connecting pressure 5 bar [72.5 psi] !

10. Do not bend or kink the hoses (Fig. 6)!
11. Connect the dust extraction hose to the extraction tube (Fig. 7).

Do not connect the unit to the mains power supply yet!

4.2 Fluorescent Lamp Installation

1. Remove the plastic sheet from both sides of the lamp cover (Fig. 8).
 2. Push the lamp cover over the fluorescent tube (Fig. 9).
 3. Insert the tube horizontally (Fig. 10).
- >>> **Please observe the safety instructions (Sec. 3)!**

4.3 Starting the Unit

1. Connect the mains power cord to the sandblasting unit (Fig. 11), then plug the power cord into the wall outlet (Fig. 12).
2. Switch the unit on (Fig. 13).

4.4 Adjusting the Operating Pressure

Caution: Never activate the pressure regulator in the maintenance unit. The regulator has been factory-set for a pressure of 6 bar to ensure optimum pinch valve function.

1. Select the sandblasting tank (Fig. 14).
2. Activate the foot switch (Fig. 15) and adjust a blasting pressure of 1 - 6 bar (14.5 - 87 psi) (Fig. 16). The pressure is displayed on the manometer (Fig. 16a). The pressure adjustment applies to all sandblasting tanks.

4.5 Filling the Sandblasting Tanks

Caution: Never press the foot switch during filling. Before closing the tank, be sure to clean the threads in the cover and the tank to remove any abrasive residue.

1. Release the pressure from the tank (Fig. 17).
a = tank under pressure, b = pressure released.
2. Unscrew the cover (Fig. 18).
3. Add abrasive up to the maximum fill level (the point where the tank jacket thickens just below the thread) (Fig. 19+20). Make sure the abrasive being used is compatible with the sandblasting tank.
4. Reinstall the tank cover (Fig. 21).

Caution: Do not cross-thread the cover.

Always use only clean, dry abrasive of the appropriate grain size (refer to the accessories).
The sandblasting unit is now operational.
A slight sound or rushing air while the unit is operating is quite normal and does not indicate a fault.

5. Cleaning / Maintenance

5.1 Cleaning the Interior

- Use only solvent-free cleansers (e.g., soapy water).
- Remove the grate and vacuum the blasting chamber (Fig. 22+23).
- Wipe out the tanks and lids only with a dry cloth!

5.2 Condensation

1. Inspect the water separator.
2. Drain any water off through the valve (Fig. 24).

5.3 Blasting Nozzle Replacement

1. Refer to insert no. 21-9162 for installation.
2. Inspect for abrasive residue.
3. Inspect the seal ring position.

5.4 View Port Replacement

Please refer to the installation instructions provided with the new pane (insert no. 21-9811).

5.5 Sandblasting Hose Replacement

1. Disconnect the sandblasting unit from the compressed air supply (Fig. 5,4,3).
2. Refer to insert no. 21-9162 for the installation instructions.

5.6 Fluorescent Tube Replacement

1. Pull the fluorescent tube horizontally out of its socket (Fig. 10).
2. Remove the cover (Fig. 9)
3. If possible, vacuum out the socket.
4. Push the cover onto the new fluorescent tube (Fig. 9).
5. Insert the new fluorescent tube horizontally into the socket (Fig. 10).

Please use only Renfert original replacement fluorescent tube sets (no. 92900-0002), Osram Dulux S/E 11W / 21-840, Philips PL-S 11W / 840 / 4P, or Sylvia CF-S 11W / 840 / fluorescent tubes.

>>> **Please observe the safety instructions (Sec. 3)!**

6. Spare Parts

Please refer to the enclosed spare parts list for all replacement parts.

7. Warranty

Provided the unit is properly used, Renfert warrants all parts of the *basic quattro IS* for a **period of 3 years**. In case of any claims for warranty original dealers' invoice is required. Components subject to natural wear (e.g., blasting nozzles, hoses, fluorescent tubes, hose connections, view panes, filter materials, etc.) are excluded from this warranty. The warranty is voided in case of improper use; failure to observe the operating, cleaning, maintenance, and connection instructions; in case of independent repairs or repairs by unauthorized personnel; if spare parts from other manufacturers are employed, or; in case of unusual influences or influences not in compliance with the utilization instructions. Warranty service shall not extend the original warranty.

8. Technical Specifications

Working pressure:	1 - 6 bar [14.5 - 87 psi]
Connecting pressure, external:	5 - 8 bar [72.5 - 116 psi]
Air consumption:	120 l/min. at 6 bar [87 psi]
Voltage:	230/240 V, 50/60 Hz 90-120 V, 50/60 Hz

Lamp power:	11 W
Lamp power consumption:	12.3 W (230/240 V) 9 W (90-120 V)
Cable length:	2 m [78,74 inch]
Ø hose connection:	interior: 4 mm [0,16 inch] exterior: 6 mm [0,24 inch]
Ø size of pipe union for external extraction unit:	interior: 35 mm [1,38 inch] exterior: 40 mm [1,58 inch]
Dimensions 4-tank version (WxHxD):	425 x 305 x 560 mm [16.74 x 12.02 x 22.06 inch]
Blasting chamber volume:	21 l
Tank capacity:	1000 ml each
Weight 2-tank version (empty):	approx. 12 kg
Noise level according to DIN 45635-01-KL3:	< 72dB(A)

9. Shipping Notes

- Empty all sandblasting tanks.
 - Request a Renfert shipping box (Tel.: +49 7731 8208-61).
 - If a Renfert shipping box is not used, always disassemble any additional tanks (otherwise we can assume no liability for shipping damage).
- The return form can be downloaded from the „Advice“ section on the Renfert website at www.renfert.com.

10. Standard Delivery

- 1 *basic quattro IS*
- 1 Foot switch
- 1 Connection set
- 1 Fluorescent tube, with lamp cover
- 1 Accessories set
- 2 Abrasive samples
- 1 Set of Operating Instructions, with attachment
- 1 Spare parts list

11. Delivery Versions

- No. 2955-0000 *basic quattro IS* basic unit, 2 tanks, 230/240 V, 50-60 Hz
1 x 25-70 µm + 1 x 70-250 µm
- No. 2955-1000 *basic quattro IS* basic unit, 2 tanks, 90-120 V, 50-60 Hz
1 x 25-70 µm + 1 x 70-250 µm

12. Accessories

Additional tanks

No. 2955-0050	25-70 µm	Right / blue
No. 2955-0051	25-70 µm	Left / red
No. 2955-0250	70-250 µm	Right / blue
No. 2955-0251	70-250 µm	Left / red

Protective mesh

- No. 2960-0003 Protective mesh for view port

Abrasives

Cobra aluminium oxide

- No. 1594-1105 25 µm [500 mesh], white 5 kg canister
- No. 1594-1205 50 µm [270 mesh], white 5 kg canister
- No. 1594-2220 50 µm [270 mesh], white 20 kg bucket
- No. 1584-1005 90 µm [170 mesh], white 5 kg canister
- No. 1583-1005 10 µm [150 mesh], white 5 kg canister
- No. 1583-1020 110 µm [150 mesh], white 20 kg bucket
- No. 1587-1005 125 µm [115 mesh], pink 5 kg canister

- No. 1587-1020 125 µm [115 mesh], pink 20 kg bucket
- No. 1585-1005 250 µm [60 mesh], white 5 kg canister
- No. 1585-1020 250 µm [60 mesh], white 20 kg bucket

Rolloblast glass beads

- No. 1594-1305 50 µm [400-200 mesh] 5 kg canister
- No. 1594-2312 50 µm [400-200 mesh] 12.5 kg canister
- No. 1589-1005 100 µm [170-100 mesh] 5 kg canister

13. Troubleshooting Guide

Problem	Possible cause	Corrective action
No air and no abrasive.	<ul style="list-style-type: none"> • Blasting nozzle blocked at the hand-piece. • Foot switch connecting lines kinked. • Switch set to an intermediate position. 	<ul style="list-style-type: none"> • Unscrew the nozzle from the hand-piece and blow it out. • With the nozzle removed, blow out the blasting hose by activating the foot pedal. • Inspect/correct the connection lines. • Turn the switch until you feel it click into place.
Only air, no or very little abrasive.	<ul style="list-style-type: none"> • Metering nozzle (centre nozzle) blocked at the tank. • Mixing chamber filter combination blocked. • Abrasive contains impurities or is moist. 	<ul style="list-style-type: none"> • Unscrew the tank cover. • Seal the top of the riser pipe (blue hose) with a piece of insulation or Sello tape. • Reinstall the cover. • Hold your finger over the nozzle on the hand-piece and repeatedly activate the foot switch. • Unscrew the tank cover and remove the tape from the riser pipe. Reinstall the cover and continue normal sandblasting. <p>Install new filter sets: à Part no. 90002-8889 25-70 µm à Part no. 90002-8888 70-250 µm (Refer to the insert for the appropriate filter set):</p> <ul style="list-style-type: none"> • Disconnect the compressed air supply to the mixing chamber. • Unscrew the plug-in connection. • Remove the old filter sets. • Run a thin (1 mm) wire through the opening in the mixing chamber. • Install new filter sets as follows (number of components according to the filter set – smooth (colour marked) side facing the mixing chamber): <ol style="list-style-type: none"> 1. Screen 2. Filter, thin 3. Filter, thick 4. Screen 5. Spacer <p>Reinstall the screw-on connections in the reverse order. Make certain there is no abrasive residue on the threads. Attention: Never operate the unit without filters! Do not reinstall used filter sets!</p> <ul style="list-style-type: none"> • Replace the abrasive.

Problem	Possible cause	Corrective action
Unit blasts continuously.	<ul style="list-style-type: none"> • Foot switch connected incorrectly. • Valve in foot switch is defective. 	<ul style="list-style-type: none"> • Inspect/correct the foot switch connection. • Replace foot switch.
Lamp does not go on.	<ul style="list-style-type: none"> • Power cord not plugged in. • Fluorescent tube defective. • Faulty electronics. • Fluorescent tube not properly seated in the socket. 	<ul style="list-style-type: none"> • Plug the power cord into the wall outlet. • Replace the fluorescent tube (refer to Sec. 5.6). • Call for service to replace the electronics, or refer to the spare parts list. • Refer to Sec. 4.2, "Fluorescent Lamp Installation".
Compressed air leaking from the foot switch.	<ul style="list-style-type: none"> • Pneumatic hoses reversed. 	<ul style="list-style-type: none"> • Reverse the connections.
Foot switch fails to switch on.	<ul style="list-style-type: none"> • No pressure at the compressed air connection. • Pneumatic hoses are kinked. 	<ul style="list-style-type: none"> • Inspect the compressed air lines and/or the compressor. • Reposition the hoses.
Quick-stop not functioning (the nozzle blasts continuously).	<ul style="list-style-type: none"> • Pneumatic connections are loose. • Sandblasting hose worn. • Insufficient pressure at the maintenance unit. • Outer pressure regulator on the maintenance unit is altered (fixing screw is loosen). 	<ul style="list-style-type: none"> • Inspect the pneumatic connections for correct seating. • Replace the sandblasting hose. • Increase the maintenance unit inlet pressure to at least 5 bar. • Turn the inner pressure regulator clockwise until stop. Activate foot pedal and set the pressure regulator on the maintenance unit at 5 bar (see manometer).
Sound of rushing air when the unit is operating.	<ul style="list-style-type: none"> • A small amount of leakage is always present at the control valves, thus allowing (a minimum) of air to escape. 	<ul style="list-style-type: none"> • This is not a fault, but instead is the result of a pressure release opening which must not be blocked.
Foot switch hoses cannot be disconnected from the unit.	<ul style="list-style-type: none"> • Push-in connection is not released (Fig. 2). 	<ul style="list-style-type: none"> • Press the push-in connection ring on the hose side firmly while pulling on the hoses.

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