making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - Polski
- FAQ - Čeština
- FAQ - 中文
- FAQ - 日本語
- FAQ - 한국어
- FAQ - اللغة العربية
- FAQ - Українська

MILLO

FAQ - frequently asked questions

Made in Germany

20-6133 30032016

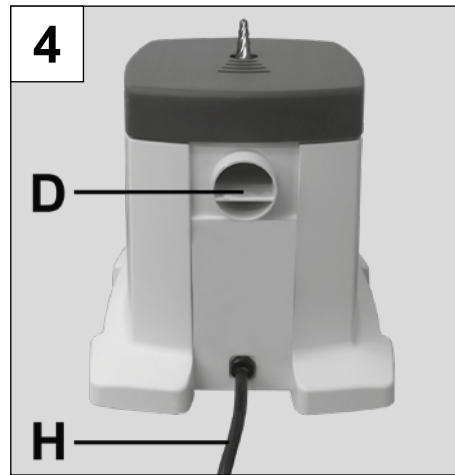
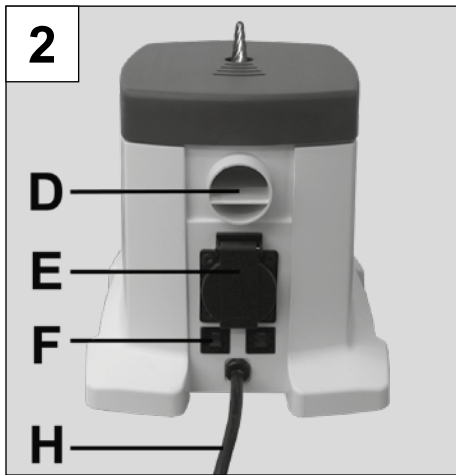
Fig. 1

FAQ - Deutsch

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FAQ - English

i For information on operating elements, see [Fig. 1](#) on page 2 of this document.

i For chapter indications, refer to the complete instruction manual on the internet.

| Problem | Cause | Remedy |
|---|---|---|
| Unit does not operate after switching it on. | <ul style="list-style-type: none"> • Mains cable not plugged in. • No electricity supply to the mains socket. • Unit connected to an extractor that is switched off. | <ul style="list-style-type: none"> • Plug in the mains cable. • Check the mains socket. • Check the extractor, refer to the extractor instructions if necessary. |
| Unit switches off during operation. | <ul style="list-style-type: none"> • Motor fuse triggered, as the motor temperature is too high. • Fuse triggered. | <ul style="list-style-type: none"> • Unplug the unit and allow it to cool for approx. 1.5 hrs. • Use an extractor (see Section 1.3). • Switch on the safety cut-out / replace fuse (see Section 3.3). • If the fuse triggers again, return the unit for repair. |
| Heavy dust accumulation on the work table. | <ul style="list-style-type: none"> • Extractor not used. • Extraction power too low. | <ul style="list-style-type: none"> • Use an extractor (see sect. 1.3). • Check the filter of the extractor. • Check that the suction hose is not bent. • Check that the suction hose is fitted securely to the connector (D, Figure 2.4). • Use a higher extraction setting. |
| Fuse triggers. | <ul style="list-style-type: none"> • Load of the connected extractor too high, adhere to the max. connected load (see technical data). | <ul style="list-style-type: none"> • Connect the extractor separately to the electricity supply. |
| Reduced removal or application of greater pressure required. | <ul style="list-style-type: none"> • Cutter blunt | <ul style="list-style-type: none"> • Change the cutter (see Section 2.5). |

| Problem | Cause | Remedy |
|--|---|---|
| Cutter slips out during operation. | <ul style="list-style-type: none"> • Cutter not inserted far enough into the clamping ring. • Clamping ring not tightened enough. | <ul style="list-style-type: none"> • Insert the cutter fully into the clamping ring. • Tighten the clamping ring further. |
| Cutter cannot be inserted. | <ul style="list-style-type: none"> • Original Renfert cutter not used. | <ul style="list-style-type: none"> • Always use original Renfert cutters. |
| Cutter cannot be removed. | <ul style="list-style-type: none"> • Clamping screw not loosened enough. • Corrosion on the cutter shank. | <ul style="list-style-type: none"> • Loosen the clamping screw further. • Return the unit for repair. |
| Clamping screw cannot be loosened. | <ul style="list-style-type: none"> • Screw overtightened. | <ul style="list-style-type: none"> • Return the unit for repair. |
| Connected extractor does not operate. | <ul style="list-style-type: none"> • Extractor not switched on. • Extractor is in automatic operation mode. • No electricity supply to the plug-in socket. | <ul style="list-style-type: none"> • Switch on the extractor. • Set the extractor to continuous operation. • Return the unit for repair. |

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