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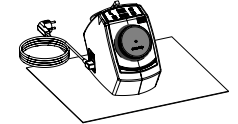
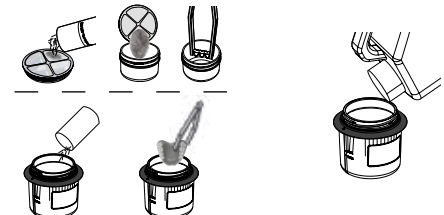
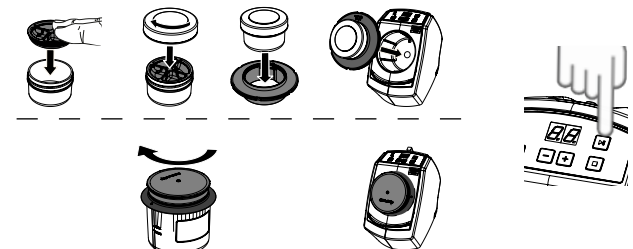
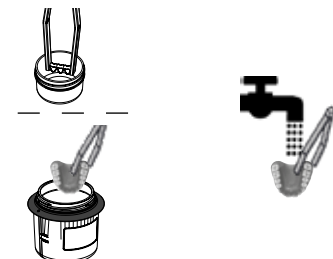
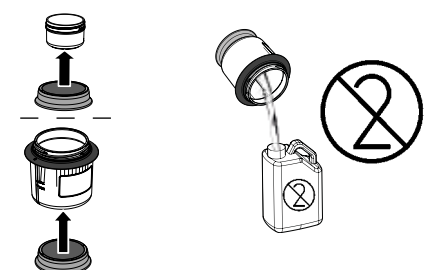
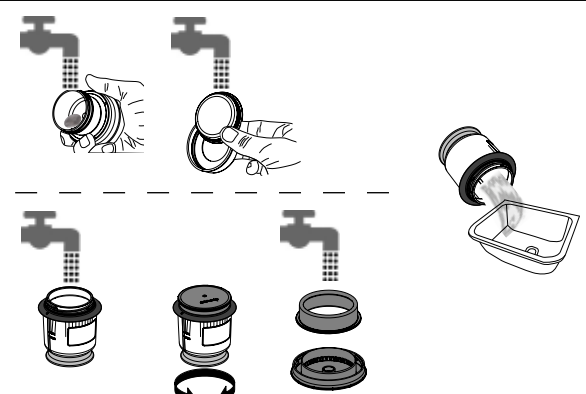
# SYMPRO Denture cleaning unit

## Important operating and safety instructions

EN

### Application Table

Please observe the following recommendations to ensure safe handling and maximum hygiene.

In addition, the detailed information provided in the operating manuals for the unit and the cleaning products should be observed.

Filling	65000450 SYMPRO disinfectable beaker	65000410 SYMPRO mini-cup
Amount of cleaning needles	75 g (1 pck.)	~ 5 g
Fill level min.	~ 200 ml (MIN marking)	10 ml
Fill level max.	MAX marking – do not exceed	Fill bowl to max. 50 %
Max. no. objects	2 complete sets of dentures or 1 misc. restoration	4 single crowns or up to 4-unit bridge

Recommended Application Parameters Contamination / Cleaning liquids			SYMPRO disinfectable beaker SYMPRO mini-cup set		Recommended application time (Depending on the degree of soiling)
			Intensity 		
<b>Disinfection procedure</b> These values apply to the disinfecting liquids listed. For other products or different concentrations, other settings may have to be used.	Bacteria and viruses	(*) Sporidicin (USA) Metri- cide OPA plus (USA) BIB forte eco 4 % (EU)	≡ High intensity		10 min.
	Bacteria and viruses	(*) Cavi Wipes (USA) Minuten Wipes (EU)	n / a		Use wipe disin- fection according to manufacturer's instructions
<b>Cleaning objects with finely detailed areas and delicate functional elements.</b> (We recommend covering especially sensitive areas)	Stubborn stains such as plaque, tartar, discoloration (tea, coffee, red wine etc.) and denture adhesive	help:ex plaque f **	— Low intensity		10 - 15 min. We recommend checking after about 5 min.
		help:ex plaque p **	—	n / a	
	Tough nicotine / tar stains and discolora- tion	help:ex discolor f	Insert into Plastic cup with lid (gray), help:ex discolor f		10 - 15 min. We recommend checking after about 5 min.
<b>Cleaning objects made par- tially of stainless steel and orthodontic restorations.</b> (We recommend covering especially sensitive areas).	Stubborn stains such as plaque, tartar, discoloration (tea, coffee, red wine etc.) and denture adhesive	help:ex plaque f **	= Medium intensity		10 - 12 min. We recommend checking after about 5 min.
		help:ex plaque p **	=	n / a	
	Tough nicotine / tar stains and discolora- tion	help:ex discolor f	Insert into Plastic cup with lid (gray), help:ex discolor f		10 - 15 min. We recommend checking after about 5 min.
<b>Cleaning objects made from NEM, total dentures, model cast frameworks and splints.</b> (We recommend covering especially sensitive areas)	Stubborn stains such as plaque, tartar, discoloration (tea, coffee, red wine etc.) and denture adhesive	help:ex plaque f **	≡ High intensity		10 - 30 min. We recommend checking after about 5 min.
		help:ex plaque p **	≡	n / a	
	Tough nicotine / tar stains and discolora- tion	help:ex discolor f	Insert into Plastic cup with lid (gray), help:ex discolor f		10 - 15 min. We recommend checking after about 5 min.
<b>Temporarily cemented restorations</b>	Dental temporary cement zinc - oxide based (with / without eugenol)	temp:ex	≡ High intensity		5 - 10 min. depending on the degree of soiling

(\*) Recommendation: Sporidicin, Contec or Metricide OPA plus, Metrex

(\*\*) help:ex plaque f – ready-to-use cleaning liquid

Help:ex plaque p – cleaning powder and neutralizer

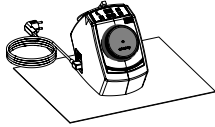
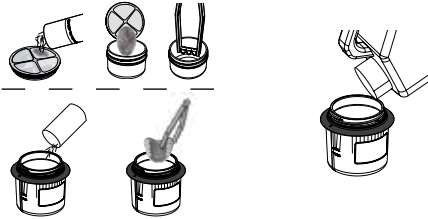
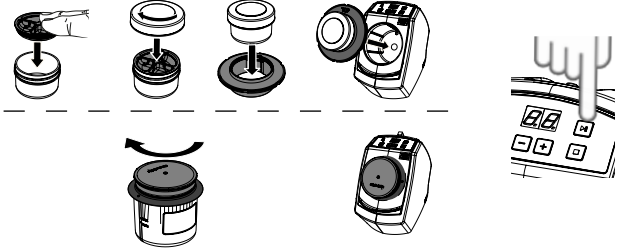
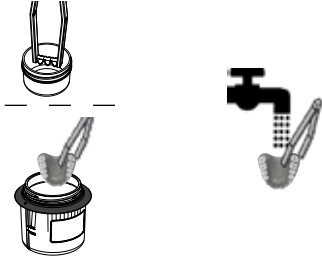
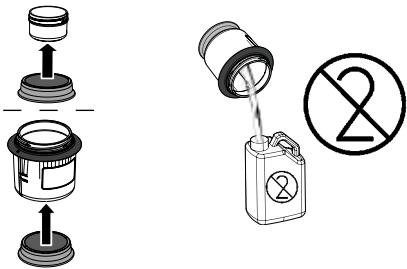
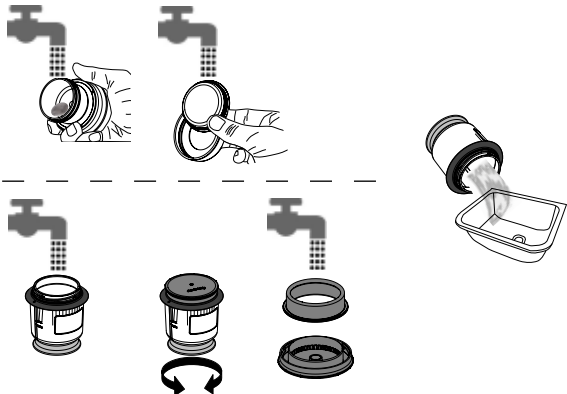
# Guide to the operation and hygienic preparation of the SYMPRO Denture Cleaning Unit system components.

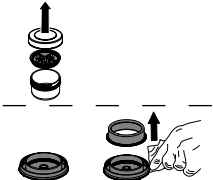
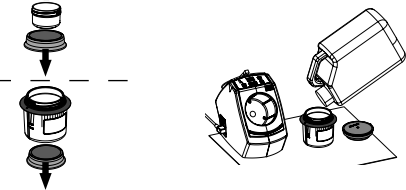
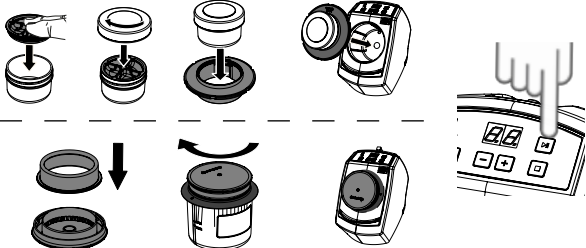
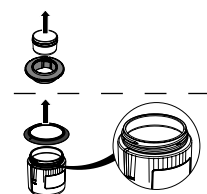
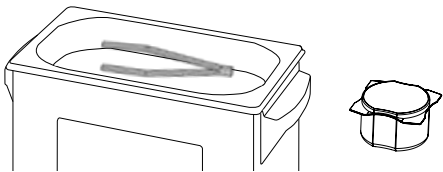
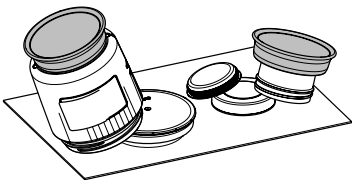
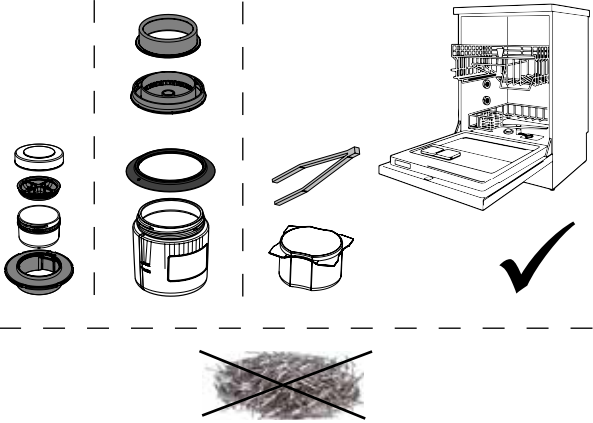
(This guide is not approved for the hygienic preparation of a dental restoration.)

## Important operating and safety instructions

Please observe the following recommendations to ensure safe handling and maximum hygiene. In addition, the detailed information provided in the operating manuals for the unit and the cleaning products should be observed.

EN

	<ul style="list-style-type: none"> <li>• Before insertion, disinfect the object to be cleaned according to the national hygiene regulations.</li> <li>• Wear personal protective equipment.</li> </ul>
<p>1.</p> 	<p><b>Installing the device</b></p> <ul style="list-style-type: none"> <li>• Place the device on an acid-resistant and non-slip work surface, if necessary, use a suitable support.</li> </ul>
<p>2.</p> 	<p><b>Filling the beaker</b></p> <ul style="list-style-type: none"> <li>• Pour the cleaning needles into the cup as shown in the Application Table.</li> <li>• Place the objects to be cleaned into the beaker.</li> <li>• Fill with cleaning fluid. (Please refer to the instructions for use of the cleaning liquid in the operating manual and safety data sheet.)</li> </ul>
<p>3.</p> 	<p><b>Close the bowl (ensure that the lid / seal is correctly in place) and insert it into the unit.</b> See recommendations on intensity and duration in the Application Table.</p>
<p>4.</p> 	<p><b>Removing the object</b></p> <ul style="list-style-type: none"> <li>• Remove the cleaned objects with tweezers.</li> <li>• Rinse carefully with water for at least 5 seconds.</li> </ul> <p><b>Tip:</b> <b>Visual inspection of the cleaned object for damage and needle deposits. Disinfect objects prior to insertion into the patient's mouth according to the national hygiene regulations!</b></p>
<p>5.</p> 	<p><b>Disposal of cleaning fluid</b></p> <ul style="list-style-type: none"> <li>• Hold the cleaning needles by placing a magnet underneath the SYMPRO cup.</li> <li>• Pour used cleaning fluid into a suitable container.</li> <li>• Dispose of according to the national regulations.</li> </ul> <p><b>Tip:</b> <b>Remove cleaning fluid residues from working surfaces immediately with a damp cloth and rub dry with a dry cloth.</b></p>
<p>6.</p> 	<p><b>Rinse the beaker and needles.</b> Fill the SYMPRO beaker up to the marked "MAX" fill level with clear water. Close the lid and clean the inside of the beaker by shaking repeatedly and intensely. When using the SYMPRO mini-cup, fill the cup to the brim under running water. Fix the cleaning needles with the help of the magnet to the underside of the SYMPRO cup. Pour the water out. This operation must be repeated at least 3 times with fresh water. Then rinse the lid and seal under running water. Check visually whether the deposits have been removed.</p>

		<p align="center"><b>Manual disinfection:</b></p>
D1		<p><b>Disinfection process – lid &amp; seal</b>  Open the cleaning beaker. Remove the silicone seal from the lid. Thoroughly rinse both objects (cover and seal) under running water. Wipe the entire lid (outer and inner surfaces including the thread) and the silicone seal (outer and inner surface) with a wipe disinfectant (*).  (*) Recommendation:  CaviWipes, Metrex</p>
D2		<p><b>Fill the cleaning beaker with disinfection liquid</b>  Remove the cleaning beaker magnets (not applicable for first time use). Fill the cleaning bowl (incl. cleaning needles) with an instrument disinfection fluid for heavy organic contamination (*). For fill quantity, see Application Table.  (*) Recommendation:  Sporicidin, Contec / Metricide OPA plus, Metrex</p>
D3		<p><b>Disinfection process – Cleaning the needles &amp; inside the beaker</b>  Replace the disinfected silicone seal back into the lid or in the case of the SYMPRO mini-cup, back into the cup. On the control panel, set a runtime of at least 10 minutes for higher intensity (*) and start the disinfection process.  (*) These values apply to the disinfection fluids recommended in D2 and in the Application Table. Other settings may be required for other types of liquid or concentrations.</p>
D4		<p><b>Disposal of disinfection liquid</b>  After the end of the program, remove and open the cleaning beaker, keep the cleaning needles with the help of the magnet, on the bottom of the cleaning beaker, pour out the disinfectant and dispose of according to the manufacturer's instructions.</p>
D5		<p><b>Disinfection process – Cleaning the outside of the beaker and &amp; sound protection ring</b>  Remove magnets. Remove the silicone sound-proof ring / adaptor ring from the cleaning bowl. Wipe the surface of the cleaning bowl (incl. thread and sound-proof ring / adapter ring) with a standard wipe disinfectant (*).  (*) Recommendation:  CaviWipes, Metrex</p>
D6		<p><b>Disinfection process – tweezers / gray cup</b>  After use, place the tweezers and vessel into an instrument bath with commercially available highlevel instrument disinfection liquid (*) for a high degree of organic soling and immerse according to the manufacturer's instructions.  (*) Recommendation:  Sporicidin, Contec / Metricide OPA plus, Metrex</p>
D7		<p><b>Next cleaning process / Storage</b>  The cleaning beaker and cleaning needles are now ready for use again. If the next use is not carried out immediately, the cleaning needles must be held on the underside of the cleaning beaker with the help of the magnet. Then, place the cleaning beaker upside down on its lid so that it can drip dry. It is advisable to place a paper towel underneath to absorb the moisture.</p>
		<p align="center"><b>Machine disinfection:</b></p>
D8		<p><b>Machine thermidisinfection of the cleaning beakers (incl. lid WITHOUT needles):</b>  The cleaning bowls, lids, silicone seals, sound-proof ring/adaptor ring, and tweezers can be thermally cleaned and disinfected in a certified cleaning and disinfecting device, according to the "Hygiene requirements for the treatment of medical Devices" (*). For this purpose, all listed components must be placed in the cleaning and disinfecting unit in a dismantled state.  <i><b>i</b> Take care when placing light parts appropriately (e.g. in the instrument basket). The cleaning needles must only be disinfected manually! Observe points D3 to D5 and store the disinfected cleaning needles in a suitable container (for example cleaning cups, spare parts) only for the duration of the cleaning process. Finally, for longer periods of storage, see point D7.</i>  (*) Recommendation of the Commission for Hospital Hygiene and Infection Prevention (KRINKO) at the Robert Koch Institute (RKI) and the Federal Institute for Drugs and Medical Devices (BfArM)</p>

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