making work easy

- de
- en
- fr
- it
- es
- pt
- tr
- ru
- pl
- zh
- ja
- ko
- ar

ORTHO guide Set MT premium

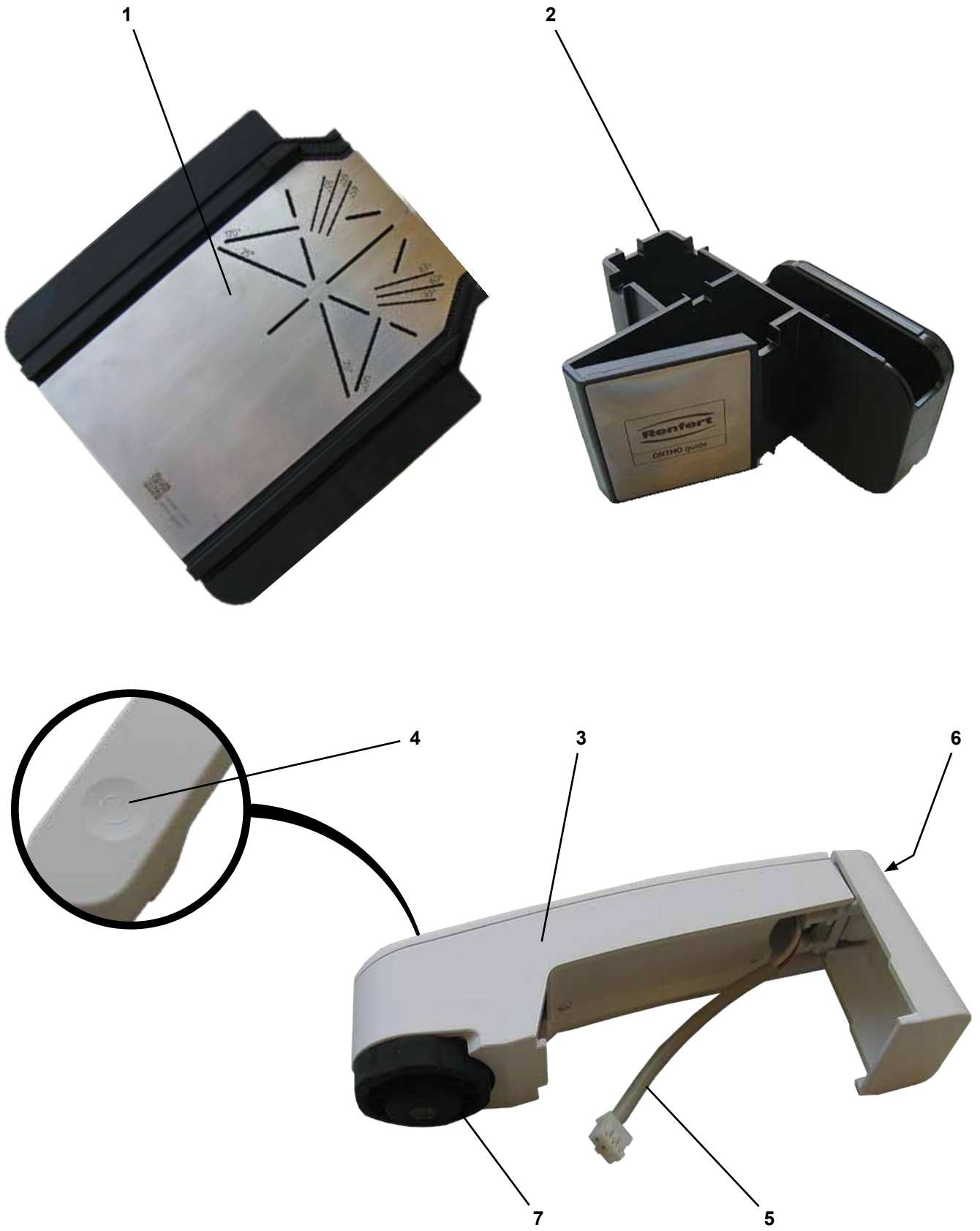
Fig. 1

1 Einleitung

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1 Introduction

These instructions describe the assembly and operation of the MT premium ORTHO guide set.

1.1 Symbols used

Symbols with the following meanings are used in these instructions or on the equipment:

Danger

Immediate danger of injury. Consult accompanying documents!

Electrical voltage

Danger due to electrical voltage.

Laser

Do not look into the laser beam.

Caution

Risk of damage to the equipment if the instruction is not followed.

Notice

Provides the operator with useful information that eases use.

2 Safety

2.1 Intended use

The MT premium ORTHO guide set can be used to extend the MT premium trimmer with an orthodontic trimmer table and a laser guide.

2.2 Improper use

The guide light may only be used in combination with the trimmer and only when attached to the trimmer as described in these instructions.

It must not be detached and used for other purposes.

2.3 Ambient conditions for safe operation

The ambient conditions for the trimmer shall apply.

2.4 Hazard and warning information

2.4.1 General information

- ▶ If the unit is not used in accordance with the supplied operating instructions, the intended safety can no longer be guaranteed.
- ▶ Disconnect the unit from the mains power supply before carrying out work on the electrical parts.
- ▶ Defective equipment must be taken out of service immediately. Disconnect the power plug and secure the equipment against further use. Send the equipment in for repair!
- ▶ Information on REACH and SVHC is available on our website at www.renfert.com, in the Support area.

2.4.2 Specific information

- ▶ **Caution! Laser beam! Class 1 laser! Do not look directly into the beam! Do not look into beams that are reflected on shiny surfaces.**
- ▶ **Only use the guide light when it is attached.**

2.5 Authorized persons

Operation and maintenance of the equipment may only be performed by trained personnel.

Any repairs not described in these operating instructions may only be performed by a qualified electrician.

2.6 Disclaimer

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- ▶ **The product is employed for any purposes other than those specified in the operating instructions.**
- ▶ **The product is altered in any way other than those alterations described in the operating instructions.**
- ▶ **The product is not repaired by an authorized dealer or if original Renfert spare parts are not used.**
- ▶ **The product continues to be used despite obvious safety defects or damage.**
- ▶ **The product is subjected to mechanical impact or is dropped.**

3 Product description

3.1 General description

You can extend the MT premium model trimmer using the MT premium ORTHO guide set.

The orthodontic model table and template enable precise trimming of the models.

The model can be precisely aligned using the guide light.

3.2 Assemblies and functional elements

See Fig. 1 (on inside cover)

- 1 Orthodontic model table
- 2 Template
- 3 Guide light
- 4 Sensor button
- 5 Connection cable
- 6 Clamping screw
- 7 Adjustment ring

3.3 Scope of delivery

- 1 Orthodontic model table
 - 1 Template
 - 1 Guide light
 - 1 STEP-BY-STEP instructions
 - 1 Operating Instructions
- Demonstration models, 1 x upper jaw, 1 x lower jaw

4 Startup

4.1 Assembly – orthodontic model table

- ⇒ Pull the attached model table upwards from the ball head at the front edge and remove it.
- ⇒ Turn the adjustment fork to the 90° position with the long side facing upwards (see also the operating instructions of the trimmer).
- ⇒ Insert the orthodontic model table and press it onto the ball head again.
- ⇒ Attach the template.

The black surface of the orthodontic model table shows milling marks caused by the template after a short time. The marks do not affect the functionality of the unit.

4.2 Assembly – guide light

⇒ Disconnect the power plug.


To mount the guide light you will need the trimmer Allen key. This is located on the left side of the unit behind the door.

⇒ Open the door locks.

⇒ Remove the trimmer door and place it to one side.

⇒ Remove the Allen key from the holder.

⇒ Lift off the cover strip with a small screwdriver. There is a small slot on the back of the cover strip for this purpose.

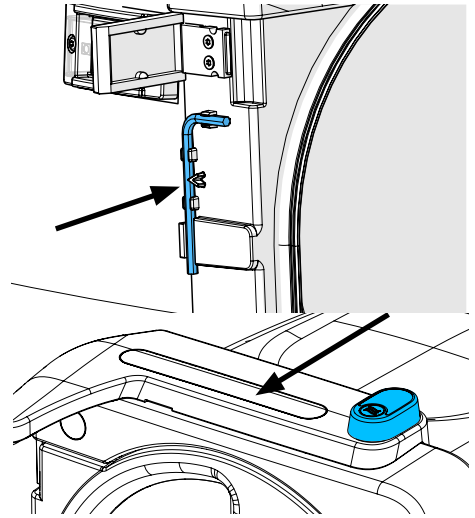
⇒ Use pointed pliers or small diagonal pliers to break open the plastic at the intended location on the cover strip.

⇒ Insert the connection cable (5, Fig. 1) in the connection point on the trimmer.

If necessary, use a small screwdriver to press the connector inwards until it clicks into place.

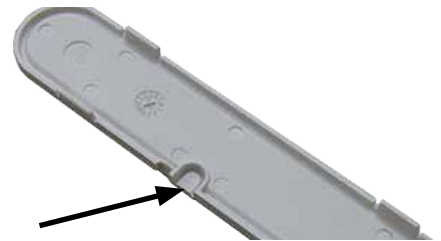⇒ Attach the cover strip again. In doing so, guide the cable through the opening created previously.

⇒ Attach the guide light. The ridges on the front and rear must engage in the slot on the unit.

The cable should not be pinched!

⇒ Tighten the clamping screw (6, Fig. 1) on the back of the guide light only slightly, as the guide light still has to be aligned.

⇒ Connect the power plug.

⇒ Switch on the guide light at the sensor button (4, Fig. 1).

⇒ Align the guide light by moving it to the right/left and turning the black adjustment wheel so that the laser beam is aligned with the median palatine raphe of the orthodontic model table.

⇒ Tighten the Allen screw finger-tight.

Do not overtighten the Allen screw!

5 Operation

The handling of the orthodontic model table and the guide light can be found in the enclosed step-by-step instructions.

There you will also find a link to user videos.

5.1 Switching on / off

The guide light can be switched on as soon as the power plug of the trimmer is connected.

It is switched on and off using the sensor button (4, Fig. 1).

The trimmer does not need to be running for this purpose.

6 Cleaning / Maintenance

6.1 Orthodontic model table and template

The orthodontic model table can be cleaned in the same way as the standard trimmer model table.

⇒ Remove the template.

⇒ Pull the model table upwards from the ball head at the front edge.

⇒ Clean the model table and the template using a brush as appropriate under running water.

Do not use any objects with sharp edges or solvent-based / abrasive cleaning agents. Never use cleaning agents containing solvents or strong acids (e.g. descaling agents).

6.2 Guide light

Only wipe the housing of the guide light with a damp cloth.

Do not use any objects with sharp edges or solvent-based / abrasive cleaning agents.

7 Troubleshooting

Errors	Cause	Corrective action
The guide light cannot be switched on.	• The power plug of the trimmer is not connected.	• Connect the power plug.
The guide light does not light up although the trimmer is running.	• The guide light is switched on/off separately, independently of the trimmer motor.	• Switch on the guide light at the sensor button (4, Fig. 1).

8 Technical data

	Guide light
Supply voltage / VDC:	5
Current consumption / mA:	30
Laser: - Laser class: - Wavelength / nm: - Output power P ₀ , max. / mW:	1 650 5
Dimensions (width x height x depth) / mm [inch]:	25 x 42 x 115 [0.98 x 1.65 x 4.53]
Weight / g [oz]:	40 [1.41]

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany